


epoc
How to Run a Sample

Switch On and Login

1. Turn on EPOC Reader (Base Unit) & EPOC Host (PDA)
2. Login to Host by selecting the keyboard at bottom right of main screen using the stylus pen provided. Enter initials in User ID & leave Password blank then select 'Login'

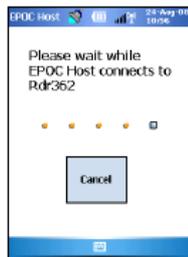

Start the Test

3. Remove card from pouch and turn so blue arrow is facing upwards
4. Using both thumbs firmly slide card into the Reader in a smooth continuous motion until it clicks into place, calibration will commence- duration 170 secs. If the card is not inserted correctly 'Unable to read barcode' will display. In this case remove the card and insert again
5. Using the tabs enter patient ID, species (required) & additional patient information
6. Take venous or arterial whole blood sample from patient- minimum 100µl*. Arterial sample required for accurate PO₂ & sO₂ results (*alternatively take blood sample prior to point 4 if preferred)
7. When EPOC displays 'Calibration Finished. Insert Sample...' introduce sample within 7 ½ minutes
8. Hold syringe barrel vertically
9. Place syringe into blood sample entry port and rotate ¼ turn to secure
10. Steadily depress syringe plunger in a single smooth continuous motion until audible beep sounds
11. Read results in 30 seconds. To print, select 
12. Remove card and discard in clinical waste



Please visit our YouTube Channel for the epoc Demonstration Video
www.youtube.com/woodleyequipment

If you require assistance please contact your Woodley Equipment Area Account Manager or Woodley Technical Support Dept. on +44 (0)1204 669033

Tel: +44 (0)1204 669033
Fax: +44 (0)1204 669034
Email: sales@woodleyequipment.com
Web www.woodleyequipment.com

WOODLEY
EQUIPMENT COMPANY LTD.