
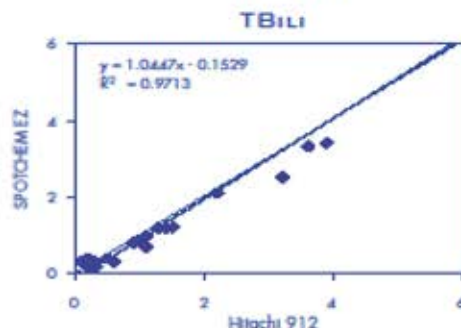
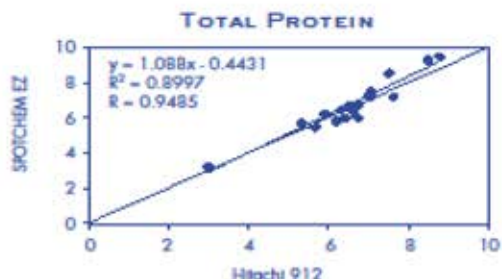
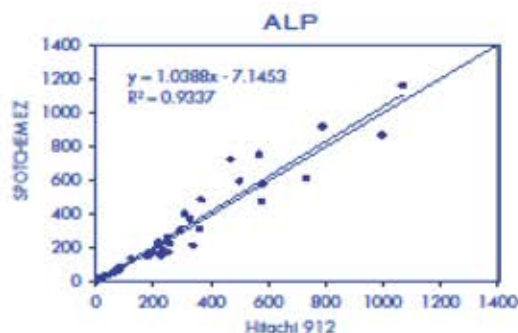
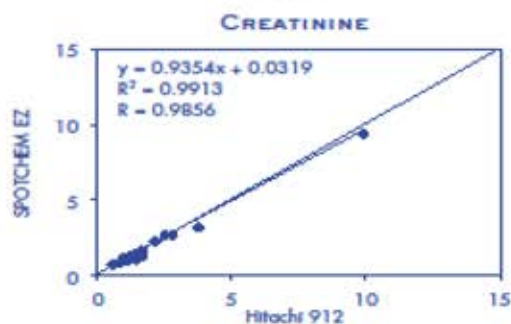
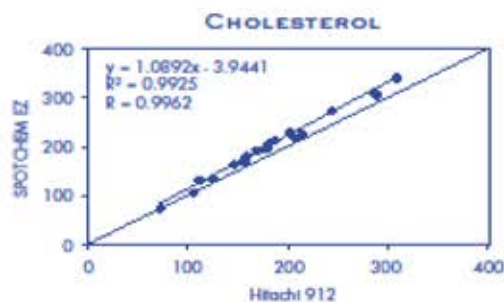
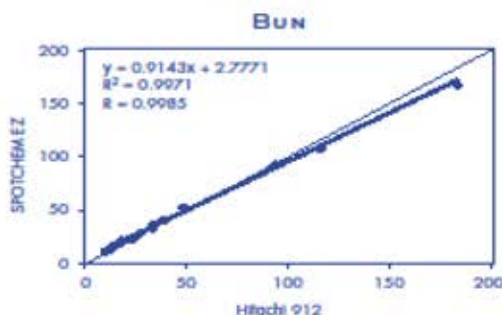
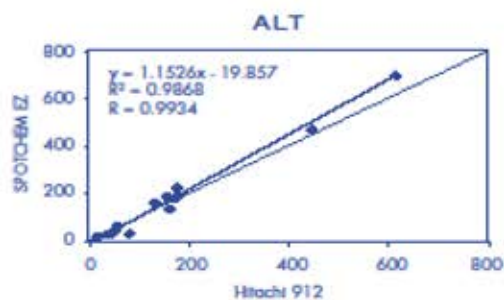


Arkray SpotChem EZ

Veterinary Clinical Chemistry Analyser

CORRELATION STUDIES

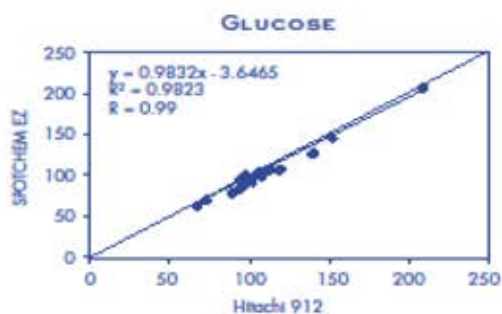

Tel: +44 (0)1204 669033


Fax: +44 (0)1204 669034

Email: sales@woodleyequipment.com


Web: www.woodleyequipment.com

WOODLEY
EQUIPMENT COMPANY LTD.

Calcium

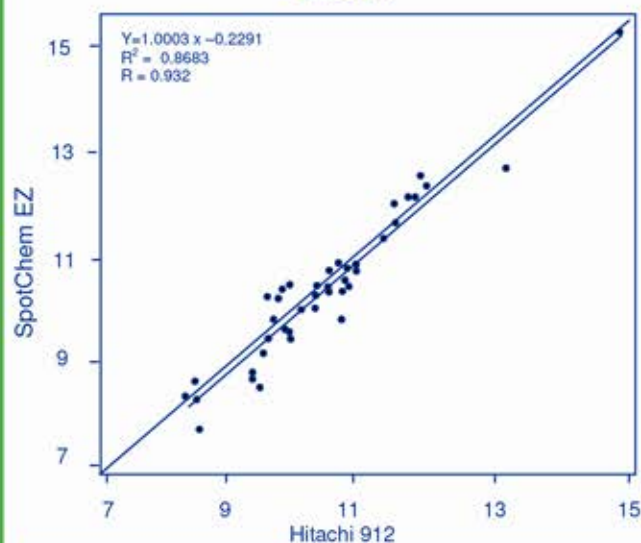

Phosphorus

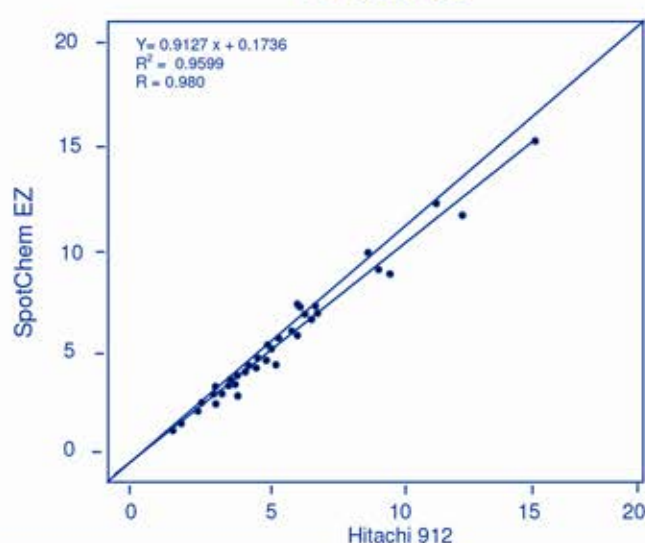

Precision Expectations

Analyte	Tested Mean	SD	Expected Min	Expected Max	CV
BUN	17	0.55	15.9	18.1	3.1
Creatinine	1.1	0.06	1.0	1.2	5.4
Glucose	94	2.8	88.4	99.6	3.0
Total Protein	7.6	0.2	7.2	8.0	2.7
Albumin	3.6	0.1	3.4	3.8	2.7
ALT	24	1	22	26	4.4
ALP	102	5.2	92	112	5.1
Calcium	9.3	0.3	8.7	9.9	3.4
Bilirubin	1.2	0.05	1.1	1.3	4.5
Phosphorus	3.6	0.09	3.4	3.8	2.4
Amylase	262	9	244	280	3.4
Cholesterol	254	5.7	243	265	2.2

PRECISION EXPECTATIONS- the same test is repeated using the same sample 10-20 times. During these tests 95% of the results are expected to fall within the MIN and MAX

SPOTCHEM is a trademark of Arkray, Inc. HESKA is a registered trademark of Heska Corporation.

Tel: +44 (0)1204 669033

Fax: +44 (0)1204 669034

Email: sales@woodleyequipment.com

Web www.woodleyequipment.com

WOODLEY
EQUIPMENT COMPANY LTD.

OLD STATION PARK BUILDINGS . ST. JOHNS STREET . HORWICH . BOLTON . LANCASHIRE . BL6 7NY