

RapidBac™ Vet

In-Practice UTI Test



TEST FOR UTI

IN UNDER 20 MINUTES.

ENABLES EARLY TREATMENT.



"PASSIONATE ABOUT LABORATORY DIAGNOSTICS"

WOODLEY
EQUIPMENT COMPANY LTD.

RapidBac™ Vet

In-Practice UTI Test

RapidBac™ Vet is a lateral flow immunoassay for the detection of Gram-positive and Gram-negative bacteria in Dog and Cat Urine.


Combine with Flexicult® Vet for complete In-Practice Urinalysis

FEATURES

- Test immediately in-practice
- Results in less than 20 minutes
- Easy to use
- Room temperature storage
- No special apparatus required

SIMPLE TEST PROCEDURE

1. Remove contents from pouch
2. Add Assay Buffer to test vial
3. After 2 minutes, add the urine sample and mix gently
4. Place the test strip into the test vial with arrows pointing down
5. Leave for 10 minutes, then remove test strip from vial and read results

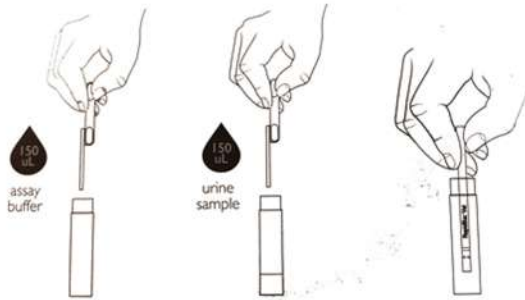

SENSITIVITY AND SPECIFICITY

	Sensitivity	Specificity	Reproducibility
Dog	97.4%	98.8%	100%
Cat	90.9%	96.6%	100%




INTERPRETING RESULTS

Positive Result

Any colour development at Line 1 or Line 2 indicates the presence of bacteria in the sample.

- If both Line 1 and Line 2 are present, a Gram-negative organism has been detected.
- If Line 2 is present and Line 1 is absent, then a Gram-positive organism has likely been detected.

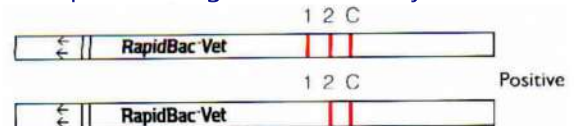

Negative Result

If only the Control Line develops colour - no bacteria detected.



Invalid Result

If the Control Line does not develop colour, the result is invalid - repeat test.

+44 (0) 1204 669033 (Option 1)

sales@woodleyequipment.com

www.woodleyequipment.com

Old Station Park Buildings
St. John Street
Horwich
Bolton
Lancashire
BL6 7NY
UK

**"PASSIONATE ABOUT
LABORATORY DIAGNOSTICS"**

**WOODLEY
EQUIPMENT COMPANY LTD.**