





Quick User Guide

Prior to first use please read the full instruction manual


Testing a Sample


1. Insert species code strip into the test port. The g-Pet PLUS+ will turn on. Press ▲ or ▼ to select species;

 (Cat)

 (Dog)


Hor (Horse)
2. Ensure the species code matches the code on the test strip vial & press **M** to save. Remove the species strip.
3. Remove a test strip from the vial & re-cap immediately (store at room temperature).
4. Insert test strip into the test port, white side facing up. The g-Pet PLUS+ will turn on automatically.
5. The meter will memorise the previously selected species. Ensure the species code matches the test strip vial.
6. Press ▲ or ▼ to select the meal indicator;

 (Pre-meal)

 (Post-meal)

No selection
7. The g-Pet PLUS+ will display a flashing ● icon when ready to test a sample.
8. Sample whole blood using an insulin syringe or the lancet provided.
9. Discard the first drop of blood & touch the tip of the test strip to the fresh blood drop until the meter beeps.
10. After 5 seconds the result will be displayed & saved to the memory.
11. Discard the test strip into clinical waste by pressing down the green strip eject button.

Testing the Quality Control Solution

1. Code the meter as instructed above. Use any of the 3 species options.
2. Insert a test strip into the test port, white side facing up.
3. The g-Pet PLUS+ will display a flashing ● icon when ready to test a sample.
4. Press & hold **M** for 2 seconds to change to Quality Control mode.  will appear on the display.
5. Mix the control solution & discard the first drop.
6. Put a drop of control solution on a clean, non-absorbent surface & touch the tip of the test strip to the drop until the meter beeps.
7. After 5 seconds the result will be displayed & saved to the memory.
8. Compare result to the relevant specified reference range on the test strip vial.
9. Discard the test strip into clinical waste by pressing down the green strip eject button.

When do you test the quality control?

- When using the g-Pet PLUS+ for the first time
- On a weekly basis to assess the performance of the g-Pet
- When using a new batch of test strips

What if the control results are outside the normal range?

- Has the test been performed correctly?
- Has the control solution expired or been stored correctly?
- Have the test strips expired or been stored correctly?
- Has the g-Pet been coded correctly?

If you require assistance please contact your Woodley Equipment Area Account Manager or Woodley Equipment Company Technical Support Dept. on 01204 669033 option 1

<p>Tel: +44 (0)1204 669033 Fax: +44 (0)1204 669034 Email: sales@woodleyequipment.com Web www.woodleyequipment.com</p>	<p>WOODLEY EQUIPMENT COMPANY LTD.</p>
--	--

OLD STATION PARK BUILDINGS . ST. JOHNS STREET . HORWICH . BOLTON . LANCASHIRE . BL6 7NY