



## How to perform Software update on InSight V-CHEM

### By USB

1. On a blank USB, create folder called 'updatesoft'
2. Copy latest software update to this folder
3. Insert USB into the USB port on the back of the analyser
4. On analyser screen, select Settings- Analyser- SW- update
5. Analyser will update, then will prompt to restart analyser.

### By WIFI

1. On analyser screen select Settings- network- scan
2. Select local WIFI and input WIFI password
3. Once analyser is connected to WIFI, go back to previous screen
4. Select Analyser-SW- update
5. Analyser will automatically download software upgrade
6. Once upgrade completed, analyser will prompt to restart.