




How to Run a Sample Using the Mythic 18

1. Use EDTA K3 anticoagulated blood collection tube
2. Fill 2/3's of the tube with blood
3. Use the touch screen to select species from main menu:



4. Enter patient ID using **A..Z** on touch screen or adjacent number keys
5. Press **VALID** key to accept patient information
6. Thoroughly mix blood on rotary mixer or by slowly inverting the tube 10 times
7. REMOVE SAMPLE CAP and place needle into sample below meniscus
8. Press the sampling trigger (below)
9. Remove sample when needle is taken into the analyser. The cycle LED changes to red
10. Results will automatically display in 60 secs once analysis is complete

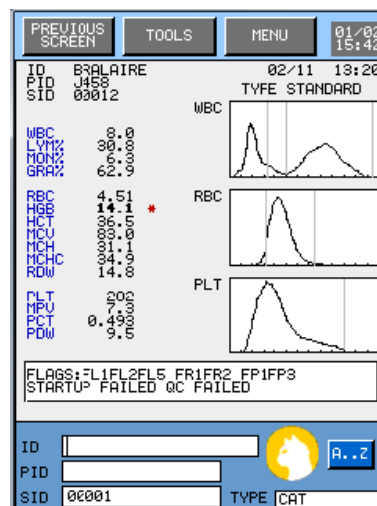

If you require assistance please contact your Woodley Equipment Area Account Manager or Woodley Technical Support Dept. on +44 (0)1204 669033

Tel: +44 (0)1204 669033
Fax: +44 (0)1204 669034
 Email: sales@woodleyequipment.com
 Web www.woodleyequipment.com

WOODLEY
EQUIPMENT COMPANY LTD.