



# InSight<sup>®</sup> Equine-IA Quick User Guide

## Setup

1. Connect the power cable to the side of the analyser.
2. Press and hold the power button for 3 seconds.
3. Allow the analyser to complete the self-test (the cartridge slot should open automatically).

## Testing Procedure

1. From the main menu screen, click “Test”.
2. Insert the ID chip in the slot at the side of the analyser.
3. Select “Read ID Card” (ID chips only need to be read when the lot changes).
4. Enter patient details as prompted.
5. Confirm the test type, sample type and ensure “Standard” test is selected.
6. Prepare the sample and test cartridge according to the test insert. Use the specified sample type and volume (10µl or 75µl). Check the insert for the specific sample type for the test being used.
7. Add the required volume (10µl or 75µl) of sample to the supplied diluent tube.
8. Mix the sample and diluent.
9. Use the 75µl pipette to add the mixed sample to the test cartridge.
10. Insert the prepared test cartridge into the cartridge slot on the front of the analyser.
11. Select “Test” on the screen to start analysis.
  - Do not touch the cartridge slot while it is moving.
  - The analyser cannot be operated during testing.

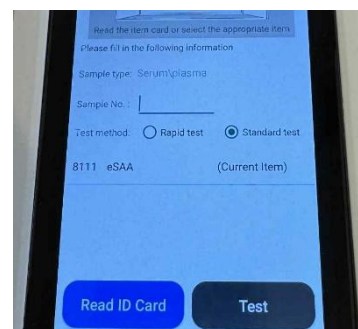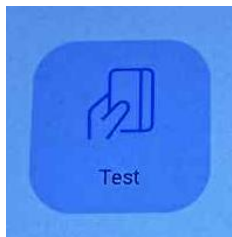

P/S-4310 Rev: 1

**Tel:** +44 (0) 1204 669033 (Option 1)

**Email:** [sales@woodleyequipment.com](mailto:sales@woodleyequipment.com)

**Web:** [www.woodleyequipment.com](http://www.woodleyequipment.com)

**WOODLEY**  
EQUIPMENT COMPANY LTD.

*“PASSIONATE ABOUT LABORATORY DIAGNOSTICS”*

## Results

12. Results are displayed automatically when testing is complete.
13. Select "Print" to print the result.
14. Select "Testing Finished" to return to the main screen.

## End of Test

15. Remove and dispose of the used cartridge and materials according to local regulations.
16. Ensure the cartridge slot has fully reset before the next test.

## Cleaning (Quick Reminder)

- Power off and unplug before cleaning.
- Wipe with a lint-free cloth.
- Use alcohol or mild detergent only.

