

# InSight Veterinary Blood Urea Test Strips

## Quick User Guide

Please read these instructions carefully before carrying out the test

Sample type: Venous or Capillary Whole Blood

Test Procedure:

1. Remove from the vial only enough test strips for immediate use and replace cap tightly
2. Apply a large drop of venous or capillary whole blood to completely cover the reagent pad
3. Set timer for 90seconds
4. After 90 seconds (**this time is critical to obtain correct results**), wash off the blood from the test strip with a stream of Distilled water from a wash bottle for 2 seconds until blood is completely washed off the test strip
5. Immediately after washing, compare the reagent pad to the colour chart below and record result. **Care must be taken to read results before colour fades.**

Blood Urea (mmol/L)



0 - 1.8



3.6 - 8.9



9.0 - 17.9



18.0 - 28.6 mmol/L