

Operator Manual

InSight

Electrolyte Analyzer
Na⁺/K⁺/Cl⁻/Ca⁺⁺/Li

- Important Information! -

These Instructions for Use contain vital warning and safety information.

This instrument is intended to be used only for the specialized purpose described in the instructions. The most important prerequisites for use, operation, and safety are explained to ensure smooth operation. No warranty or liability claims will be covered if the instrument is used in ways other than those described or if the necessary prerequisites and safety measures are not observed.

The instrument may be operated only by persons whose qualifications enable them to comply with the safety measures that are necessary during operation of the instrument.

Adjustments and maintenance performed with removed covers and connected power may be attempted only by a qualified technician who is aware of the associated dangers.

Instrument repairs are to be performed only by the manufacturer or qualified service personnel.

Operation of the instrument with solutions whose composition is not consistent with that of the original solutions can negatively affect, above all, the long term measurement accuracy. Deviations in the composition of the solutions can also decrease the service life of the electrodes.

The quality control requirements must be completed at least once daily. Because accurate measurement results depend not only on the proper functioning of the instrument, but also on a number of other factors (such as preanalytics), the results produced by the instrument should be examined by a trained expert before subsequent decisions are reached that are based on the measurement values.

Explanation:



"Caution, refer to accompanying documents".

- Important Information! -

- Operating safety information -

- The instrument has been constructed and tested according to the protective measures stipulated by EN 61010-1 / IEC 61010-1 / EN 6101010-2-101 for electrical measurement, control, IVD, and laboratory instruments and was delivered from the factory in flawless condition with regards to safety features. In order to preserve this condition and ensure safe operation, the user must respect the notices and warnings that are contained in these Instructions for Use.
- This instrument is classified under the protection class I according to EN 60000-1 / IEC 1010-1.
- The instrument meets the conditions for overvoltage category II.
- The instrument meets the conditions for contamination level 2.
- Do not operate the instrument in an explosive environment or in the vicinity of explosive anesthetic mixtures containing oxygen or nitrous oxide.
- If an object or liquid enters the internal areas of the instrument, remove the instrument from its power supply and allow an expert to check it thoroughly before using it again.
- The instrument is suitable for long-term operation indoors.

CAUTION:

- Plug the power cord into a grounded socket only. When using an extension cord, make sure it is properly grounded.
- Any rupture of the ground lead inside or outside the instrument or a loose ground connection may result in hazardous operating conditions. Intentional disconnection of the grounding is not permitted.
- The instrument is not suitable for operation with a direct current power supply.
Use only the original mains plug delivered with the instrument.

- Operating safety information -

1

1 Introduction

1.1	General notes	1-2
1.1.1	Symbols.....	1-2
	Used in the Instructions for Use	1-4
1.2	Measurement and calibration procedures.....	1-5
1.2.1	Measurement procedure.....	1-5
1.2.2	Calibration procedure.....	1-5
1.3	Measurement evaluation.....	1-5
1.4	Important safety instructions	1-6
1.4.1	Sample collection and handling.....	1-6
1.4.2	Disposal of fluids, electrodes, and instrument.....	1-6
	Disposal of the reference electrode.....	1-6
1.5	System description.....	1-7
1.5.1	Analyzer components.....	1-7
	1. Display.....	1-8
	2. Keypad.....	1-8
	3. Printer	1-8
	4. Measuring chamber	1-9
	5. Peristaltic pump.....	1-9
	6. Sample probe mechanism.....	1-9
	7. Valves	1-10
	8. ISE PACK	1-10
	9. Rear panel.....	1-11
1.6	Installation	1-12
1.6.1	Location.....	1-12
	Accessories	1-13
1.6.2	Setting up.....	1-13
1.6.3	Electrodes and measurement chamber	1-14
1.6.4	Preparing the Analyzer for Operation	1-18
	1. Selecting language.....	1-18
	2. Start-up	1-18
	3. Setting date and time	1-19
	4. Installing the ISE PACK	1-20
	5. Installing the printer paper.....	1-20
	6. Maintenance (manual).....	1-22
	7. Selecting parameter configuration.....	1-24

1.7 Shutdown.....	1-26
1. Installing shutdown plug.....	1-26
2. Installing the transport housing.....	1-27
3. Washing the lines	1-27
4. Removing the electrodes and the shutdown plug.....	1-28
5. Installing the relief clamps and removing the pump windings	1-28
6. Turning the analyzer “off”	1-29
1.8 Standby mode.....	1-30

1 Introduction

The InSight Electrolyte Analyzer is designed to quickly, accurately and efficiently conduct basic electrolyte testing in the convenience of the laboratory.

This manual will help guide you through setting up the analyzer and starting sample analysis. As the user becomes familiar with the operation of the unit, the manual is a useful reference for day-to-day routines and as a guide for maintenance and troubleshooting.



Fig. 1-1

1.1 General notes

1.1.1 Symbols



This product fulfills the requirements of the Directive 98/79/EC for in vitro diagnostic medical devices.



Lot number

Expiration date



Electrodes: install by...


The electrode must be inserted before the indicated date, but can remain in the instrument after the date.



Solutions: use by...

The solution must be completely consumed by the indicated date.

If a day is not indicated, apply the last day of the respective month.

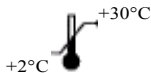

Storage temperature:

The conditions necessary to preserve the product's shelf life.



For in vitro diagnostic use



Manufacturer-according to In Vitro Diagnostic guidelines 98/79/EC



Store upright



Risk of infection (according to the standard DIN EN 61010-2-101)



Reference and/or ordering number



IMPORTANT: read and follow Instructions for Use!



Please read and follow the information on the packaging insert / instructions for use.



Serial number (model plate)

Used in the Instructions for Use



Sections marked with this symbol (see Instructions for Use) contain information that must be observed to avoid potential injuries (to patients, users and third parties)



Risk of infection!



All sections / passages that are marked with this symbol describe procedures and/or indicate conditions or dangers that could damage or lead to a malfunction in the InSight Electrolyte Analyzer, and which therefore should never be attempted.

TIP: *All sections / text locations marked with "TIP" describe safe procedures that are intended to provide the user with additional help.*

1.2 Measurement and calibration procedures

1.2.1 Measurement procedure

The InSight Electrolyte Analyzer methodology is based on the ion selective electrode (ISE) measurement to determine the measurement values (see chapter 7, section "The measurement principle").

There are six different electrodes used in the InSight Electrolyte Analyzer: sodium, potassium, chloride, ionized calcium, lithium and a reference electrode. Each electrode has an ion selective membrane that undergoes a specific reaction with the corresponding ions contained in the sample being analyzed. The membrane is an ion exchanger, reacting to the electrical charge of the ion causing a change in the membrane potential, or measuring voltage, which is built up in the film between the sample and the membrane.

A galvanic measuring chain within the electrode determines the difference between the two potential values on either side of the membrane. The galvanic chain is closed through the sample on one side by the reference electrode, reference electrolyte and the "open terminal". The membrane, inner electrolyte and inner electrode close the other side.

A difference in ion concentrations between the inner electrolyte and the sample causes an electro-chemical potential to form across the membrane of the active electrode. The potential is conducted by a highly conductive, inner electrode to an amplifier. The reference electrode is connected to ground as well as to the amplifier.

The ion concentration in the sample is then determined by using a calibration curve determined by measured points of standard solutions with precisely known ion concentrations.

1.2.2 Calibration procedure

A 2-point (non-Li configuration) or a 3-point (Li configurations) calibration is performed automatically every 4 hours in [H-READY] mode and a 1-point calibration is automatically performed after each measurement.

An automatic calibration procedure is also performed shortly after power-on or reset. A calibration cycle can also be initiated manually at times when no sample measurements are being performed.

1.3 Measurement evaluation

The validity of the test results from the InSight Electrolyte Analyzer must be carefully examined by a clinician who will take the patient's clinical condition into consideration before any decisions are reached.

In order to ensure the quality of the measurement results, complete quality control test on 3 levels (low, normal, high) after changing electrodes, test each level at least once daily or more often in accordance with local regulations.

1.4 Important safety instructions

For your own safety and the proper operation of your equipment, always follow these precautions when working with the InSight Electrolyte Analyzer:

- Keep the analyzer away from sinks and wash basins.



Do not use ammonia-based or alcohol-based cleaners, which can chemically react with plastic, on or around the analyzer.

- Always handle blood samples and collection devices with appropriate precautions.
- Use approved protective gloves to avoid direct contact with sample.
- Aseptic procedures are required when cleaning the sampling probe to avoid contamination.
- Dispose of ISE PACK according to local regulations.

1.4.1 Sample collection and handling

Universal precautions must be observed when collecting blood samples. It is recommended that all blood be handled as potentially infectious specimens capable of transmitting human immunodeficiency virus (HIV), hepatitis B virus (HBV), and other blood-borne pathogens. Proper blood collection technique must be followed in order to minimize risk to the laboratory staff. Gloves should always be worn when handling blood and other body fluids.

Please refer to NCCLS document M29-A2, "*Protection of Laboratory Workers from Occupationally Acquired Infections*", Approved Guidelines - Second Edition 2001, for further information on safe handling of these specimens. For further information, see chapter 3: "Measurement".

1.4.2 Disposal of ISE Pack, electrodes, and instrument



Dispose of used ISE Pack, electrodes and the instrument according to local regulations for biologically contaminated-hazardous waste!

Disposal of the reference electrode



This electrode contains mercury. Therefore dispose of it in accordance with the local and/or laboratory regulations for hazardous waste!

1.5 System description

1.5.1 Analyzer components



Fig 1-2

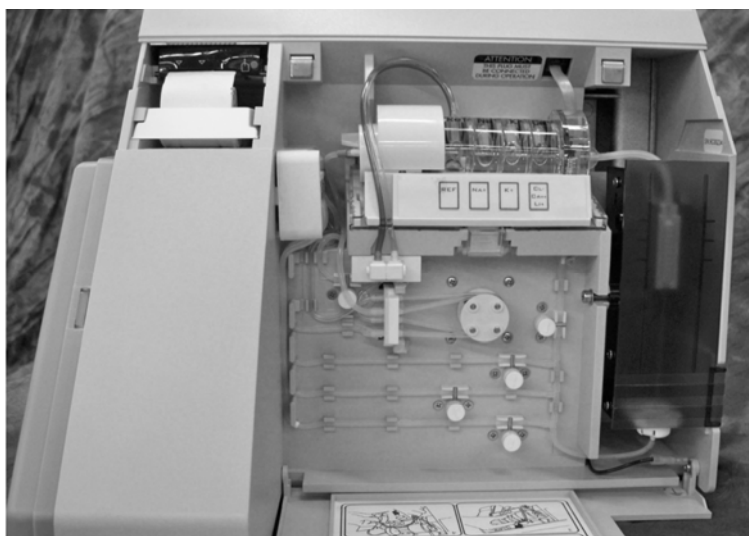


Fig 1-3

Display

The analyzer display is a two line, alphanumeric read-out which allows up to 16 characters per line, displaying the activities of the analyzer, sample results and other system information.

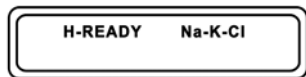


Fig. 1-4

Keypad

With the **YES** and **NO** keys all analyzer functions, including: sample measurement, data input, programming and quality control testing can be performed.



Fig. 1-5

Printer

The thermal printer uses heat-sensitive paper to output information in 16 columns. The analyzer will print measured values, calibration values, electrode voltages, and amount of liquid remaining in the ISE Pack as well as cleaning, maintenance, and error information. The unit allows convenient storage of a second roll of paper in the rear of the paper tray.



Fig. 1-6

Measuring chamber

The measuring chamber consists of the movable left locking device, four electrodes, sample sensor connector, and the measuring chamber base.



Fig. 1-7

Peristaltic pump

A peristaltic pump transports all sample and operating fluids inside the instrument.

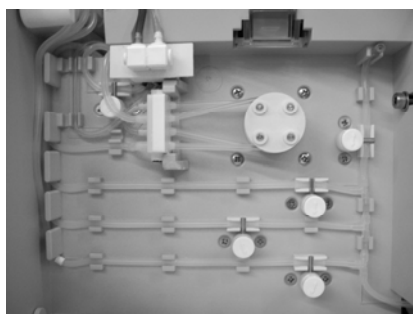


Fig. 1-8

Sample probe and cover

The sample probe mechanism is located behind the sample door at the front of the unit.



Fig. 1-9

Valves

Valves control the movement of the liquid within the analyzer.

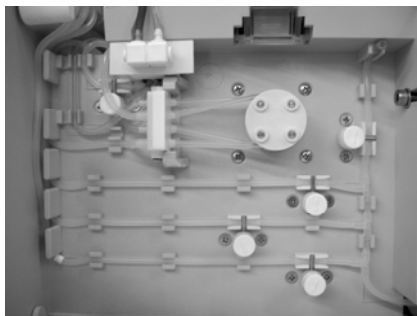


Fig. 1-10

ISE PACK

The self-contained ISE PACK is constructed to ensure that waste cannot spill out of the package.



Fig. 1-11

Rear panel

The rear panel of the unit contains a serial number plate, two USB connectors, a power switch/power receptacle module and an RS232 interface port.



Fig. 1-12

1.6 Installation

1.6.1 Location

Location is important for trouble-free operation of your analyzer. Before you begin setup, choose a site that is convenient for your sampling needs and meets the following physical requirements of the unit:

- Ambient temperature + 15°C to + 32°C
- Avoid exposure to direct sunlight, vibration and strong electromagnetic fields (electric motors, transformers, X-ray equipment, cellular phones, etc).
- Use a stable and level work surface.
- Maximum relative humidity of 85% (15 % - 85 %)
- Ample room to allow air to circulate freely around the unit, see Fig. 1-13.
- Avoid exposure to explosive gases or vapors.
- Check for correct voltage: 100 to 240 VAC, 50/60 Hz.



Fig. 1-13

After setting up the InSight Electrolyte Analyzer in a location that meets the above requirements, perform the following steps to ensure the instrument is ready for operation:

- Refer to the packing slip to check for the completeness of the shipment.
- If the shipment is incomplete, please inform your representative immediately.

If the instrument has been damaged during shipment, immediately inform the company that made the delivery. Retain all packaging materials and products as this may be needed as evidence in the event of a damage claim.



Do not remove the analyzer from the shipping carton by pulling upward on the packing materials. These packing materials do not provide strength to support the analyzer.

Accessories

The following parts are delivered as standard accessories with the InSight Electrolyte Analyzer:

- Printer paper
- Electrode Dummy
- Quad Ring, 1.78x1.02 mm
- Fuse set
- Sample probe stylet
- Syringe, 12 mL, with tapered tip
- Quick Reference Guide
- Quad ORing, 0.046"ID x 0.135" OD

You will also need a supply of lint-free tissues and disposable sample cups, which should be kept in a location convenient to the analyzer.

Optional Accessories

The following parts are optional accessories for the InSight Electrolyte Analyzer (see Section 1.6.4.1 for further information):

- Barcode Reader
- Keyboard

1.6.2 Setting up

Prior to beginning the actual installation, it is recommended that the operator read through this section thoroughly to develop an understanding of the procedures that are required.

Begin by placing the analyzer on a secure table top that allows plenty of working space and is convenient to a power connection.

Open the analyzer main door. Locate and carefully remove the five red relief clamps.

TIP: *Save the clamps for reuse to prevent damaging the tubes if the analyzer is later shut down for any reason. ("See Shutdown")*



Fig.1-14



Fig. 1-15

Slip the two pump windings around the analyzer pump rollers, making sure not to over stretch the tubing.

1.6.3 *Electrodes and measurement chamber*

The next procedure involves preparing and installing the electrodes in the measurement chamber.

- Remove the reference housing and electrodes from their protective boxes and place them on a soft, clean surface. Check to make sure each electrode has a black o-ring on the left side.

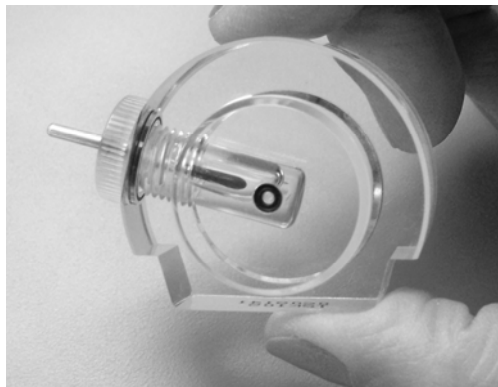


Fig.1-16

- Unscrew the transport housing from the reference electrode and check that the o-ring on the electrode is properly seated. Rinse, dry and save the transport housing for storage of the reference electrode in the event the analyzer is turned off or taken out of service.

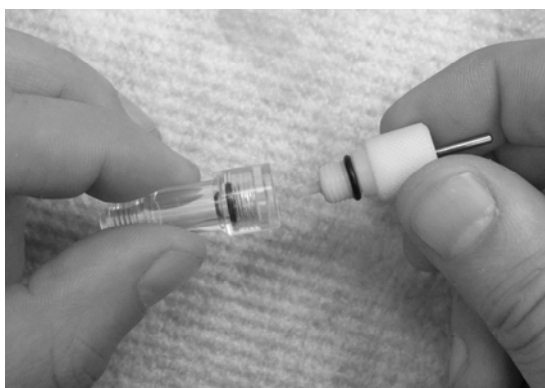


Fig.1-17

- Carefully screw the reference electrode into the reference electrode housing and place it with the other electrodes



Fig.1-18

- Slide the measurement chamber forward until it locks in the front position. (Fig 1-19)
- Unclamp the left electrode holder by moving the clamp forward. (Fig 1-20)



Fig. 1-19



Fig. 1-20

- Locate the sample sensor cable; check that an o-ring is present in the right electrode holder

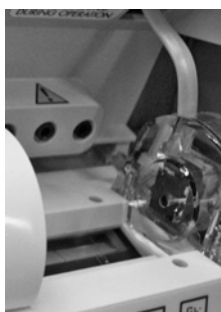


Fig. 1-21

- Now install the electrodes in the measurement chamber, beginning on the right and working to the left (the reference electrode will be installed last).

TIP: The electrode on the right may be one of the following: chloride, ionized calcium, lithium or a dummy electrode. If sodium and lithium are selected, the K^+ electrode is replaced by a dummy electrode. (See Fig. 1-26)

- Check to make sure that the lettering on the electrode corresponds with the correct electrode position on the label. Also, note that all electrodes have a lip on the bottom that rests on the flat edge of the measurement chamber to aid in proper positioning.



Fig. 1-22

- After all electrodes have been installed, close the clamp on the left electrode holder by lifting it upward until it locks in the back position. Ensure that the electrodes are properly seated by checking position of each. (Fig 1-23)



Fig. 1-23

- Slide the measurement chamber back until it snaps into position.

Tip: Electrode tray may need to be slid upwards towards the back for proper fit into electrode receptacle. Check that sample sensor is connected, connect if it is not.

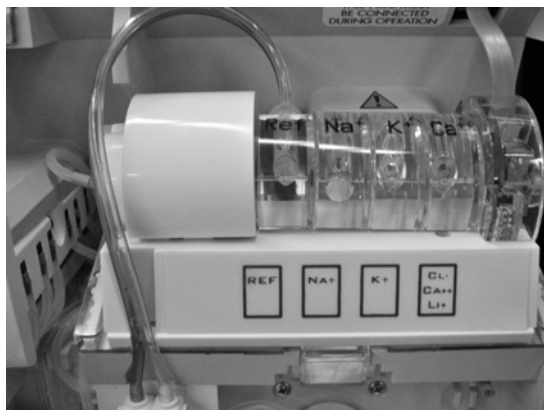


Fig. 1-24

- Plug the tubing connector of the reference housing assembly in the receptacle below the left side of the measurement chamber.

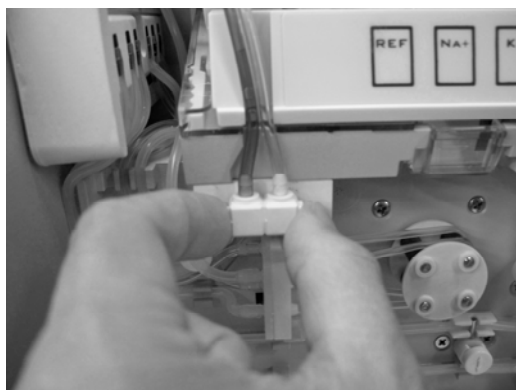


Fig. 1-25

Possible parameter configurations are the following:

		Electrode Position		
		Na	K	Ca/Cl/Li
Configuration	1	Na	K	Dummy
	2	Na	K	Cl
	3	Na	Dummy	Cl
	4	Na	K	Ca
	5	Na	Dummy	Ca
	6	Na	K	Li
	7	Na	Dummy	Li
	8	[Na]	Dummy	Li
	9	Na	Ca	Li

Fig. 1-26

For details, see section "7. Selecting parameter configuration".

1.6.4 Preparing the Analyzer for Operation

1.6.4.1 Installing optional accessories



The keyboard and barcode scanner must be plugged in prior to start-up. If the instrument is powered on before installing these items, the user must shut the analyzer off before connecting these accessories to prevent hardware failures that can occur if parts are plugged in while the unit is on.

Connectors for these items are located on the back of the instrument as shown in Fig.1-12.



Fig. 1-27



Fig. 1-28

1.6.4.2 Start-up

- Locate the power switch on the back of the unit and make sure that it is in the **OFF** (0) position.
- Plug the power cord into the power receptacle module on the back of the unit, and then plug the cord into a grounded electrical outlet (100 - 240 V, 50/60 Hz).



Fig. 1-29

- Push the power switch to the **ON** (I) position. The unit will automatically begin to operate.



Fig. 1-30

Now that the InSight Electrolyte Analyzer is functioning, begin using the keypad interface to communicate with the instrument. Use the **NO** key to make changes, the **YES** key to accept the displayed values or information.

***TIP:** After start-up, date and time should be set and the printer paper inserted.*

1.6.4.3 Selecting language

The factory-set language is English. However, to change the language:

- Press the **NO** key until **[INSTRUMENT SETTING?]** is displayed
- Select **YES**.
- Press **NO** to scroll through the menu until **[LANGUAGE SETTING?]** is shown
- Press **YES**. Upon entering this menu the language can be set to English, German, French, Spanish, Italian, or Portuguese.
- Select **YES** when the desired language is displayed on the screen.

1.6.4.4 Setting date and time

The analyzer will display a default date and time and will allow you to input the correct date and time:

DATE: 01-JUN-11

TIME: 00:00

To enter the correct date and time:

- Press **NO**, until the actual day is displayed. To accept this value, press **YES** and the cursor will move to the month.
- By keeping the **NO** key depressed, the analyzer will automatically scroll through the numbers, first slowly, then faster.
- Press **NO**, until the actual month is displayed. To accept this value, press **YES**.
- Press **NO**, until the actual year is displayed. To accept this value, press **YES**. Now the actual date will be displayed.
- Follow the same procedure to enter the current time.
- After entering the time, the analyzer will prompt **[ok?]**. Press **YES** if the date and time entered are correct, or press **NO** to make a change.
- After pressing **YES** at the last prompt, the analyzer will prompt:
[STATUS: NO PACK INSTALLED].

1.6.4.5 Installing the printer paper

TIP: The printer paper is heat sensitive on one side only. Please make sure that you insert the paper roll correctly.

The thermal printer paper supplied contains an indicator strip to alert you when the paper roll should be changed. To change the roll: See Figure 1-31

- Place a new paper roll in the paper tray and thread it into the feeder slot.



Fig. 1-31

- Press the paper feed button to completely feed the paper through the printer.



Fig.1-32

TIP: By pressing the paper advance button once, the paper will automatically advance 10 line.



To avoid damage to the printer, do not pull the paper out of the printer. Use the paper feed button to advance the paper through the printer.

1.6.4.6 Installing the ISE PACK

- Record the installation date of the ISE Pack on the label on the ISE Pack.
- Remove the protective strip and slide the ISE Pack into position on the left side of the analyzer. Save the protective strip for disposal.

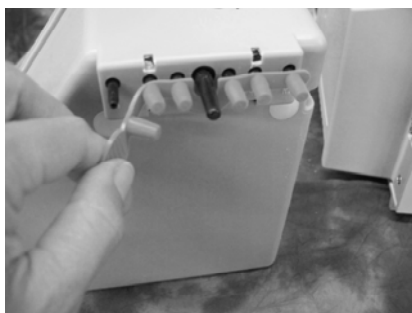


Fig. 1-33



Fig. 1-34



Once the protective strip is removed, be sure to keep the ISE Pack upright to avoid spillage.

- At the prompt [**ISE-PACK CHANGED?**], press **YES**.
- Again, at the prompt [**ARE YOU SURE?**], press **YES**.

Now perform the initial Maintenance.

1.6.4.7 Maintenance (Manual)

Prior to performing the first calibration or running the first sample, the InSight Electrolyte Analyzer needs to undergo a simple cleaning and conditioning procedure that helps ensure that the analyzer will perform properly. This procedure is called *Maintenance*. It should be performed each day the analyzer is used.

The process involves cleaning the sample path and conditioning electrodes. The bottles containing Cleaning Solution A and Electrolyte Conditioning Solution should be ready, along with a package of lint-free tissues that will be used to dry the probe.



Check expiration date on the bottles of the Cleaning Solution A and the Electrolyte Conditioning Solution.



In some cases, when the analyzer prompts you for an action and you do not respond within a set period of time, an alarm will sound and the unit will discontinue its current operation.

To start the daily maintenance:

- Pour a small amount of Cleaning Solution A into a clean container or sample cup
- Press the **NO** key, until **[MAINTENANCE?]** is displayed, and **YES** to select. Select **[CLEAN?]** by pressing **YES**.
- At the prompt **[LIFT DOOR LOAD SAMPLE]**, lift the sample door and the pump will begin to turn.



Fig. 1-35

- Dip the probe into the Cleaning Solution A until **[WIPE PROBE_SHUT SAMPLE DOOR]** is displayed. Use a lint-free tissue to remove the cleaning solution from the probe. Close the sample door.



Fig. 1-36

- While the analyzer displays **[THANK YOU]** and a brief countdown, pour a small amount of conditioning solution into a clean container sample cup.
- Answer the prompt **[CONDITION?]** by pressing **YES**.
- At the prompt **[LIFT DOOR LOAD SAMPLE]**, lift the sample door and the pump will begin to aspirate.



Fig.1-37

- Hold cup of Electrolyte Conditioning Solution under the probe, until [**WIPE PROBE_SHUT SAMPLE DOOR**] is displayed. Use a lint-free tissue to wipe the probe to remove conditioning solution.
- Close the sample door.



Fig. 1-38

- After the analyzer has displayed [**THANK YOU**] and a brief countdown, the prompt [**CONTINUE MAINTENANCE?**] can be answered with **NO**, and an automatic calibration will begin.

TIP: If the instrument is not configured as Na-K, press **NO** to stop the calibration and proceed to 1.6.4.8.



It is very important that the main door is closed during calibration, since it provides shielding from sources of electromagnetic interference.



Calibration is an automatic process. During this time, the analyzer is conducting measurement operations to ensure the accuracy of the instrument. Occasionally, additional time is required, and the analyzer displays an asterisk.

After finishing the calibration, the analyzer displays [**H-READY Na-K-CI**] and is now ready for sample analysis.

1.6.4.8 Selecting parameter configuration

After completion of the daily maintenance procedure, the analyzer starts an automatic calibration for sodium and potassium.

To select a different parameter configuration, interrupt the calibration by pressing **NO** shortly after it starts.

- Continue pressing **NO**, until [**OPERATOR SETTINGS?**] is displayed. Enter **YES**.
- Press **NO**, until the prompt [**CHOOSE PARAMETERS?**] is displayed, see Fig. 1-26.
- Enter **YES** and the current configuration will be displayed. The default configuration is the following:

SEL. PARAMETER:
[Na] [K] [] ok?

- Press **NO** until the desired configuration is displayed. Enter **YES** to accept this selection.

TIP: *If [(Na)] [] [Li] is selected, only lithium is reported. Install the sodium electrode and calibrate for lithium.*

The InSight Electrolyte Analyzer is now ready to operate.

1.7 Shutdown

A complete shutdown of the analyzer may be necessary to prepare the analyzer for shipping or in case the analyzer is not being used for an extended period of time. For this procedure, you will need a special shutdown kit.



Never attempt to turn the power off for an extended period of time without performing a complete shutdown of the analyzer.

In case the analyzer is not being used for several days only, it is not recommended to perform a complete shutdown, but to put the analyzer in Standby Mode.

To perform the shutdown, you will need the following items:

- One shutdown plug (supplied with shutdown kit).
- Five solenoid relief clamps (supplied with analyzer and shutdown kit).
- One reference electrode transport housing (supplied with shutdown kit).
- Two containers, one filled with at least 100 mL of water, the other one empty.
- Protective strip for ISE Pack (supplied with shutdown kit).

Starting at the [**H-READY**] display, do the following:

- Press **NO** until [**OPERATOR FUNCTIONS?**] is displayed. Enter **YES**.
- Select [**ACTIVATE SHUTDOWN?**] and enter **YES**.
- Confirm [**ARE YOU SURE?**] by entering **YES**.

1.7.1 Installing shutdown plug

The analyzer prompts [**WATER & PLUG INSTALLED?**].

- Remove the ISE Pack and push the protective strip firmly onto the ISE Pack connector.
- Fill one container of the shutdown kit about halfway with distilled water.
- Insert the shutdown plug carefully into the ISE Pack receptacle.
- Place the blue marked line into the disposable container filled with water
- Place the red marked line into the empty container.
- Begin the tubing wash procedure by pressing **YES**.



Fig. 1-39

1.7.2 *Installing the transport housing*

The analyzer will prompt [**REF PROTECTIVE HSG INSTALLED?**].

- Pull the electrode holder forward and remove the reference electrode assembly.
- Carefully unscrew the reference electrode from the reference electrode housing.
- Temporarily place the reference electrode on a clean, soft cloth.
- Pour the remaining reference solution in the reference housing into a transport housing.
- Carefully screw the reference electrode into the filled transport housing.
- The second transport housing is screwed into the reference housing.
- Place the reference housing back into the electrode holder and move the lever backwards, making sure all electrodes are seated properly. The electrode holder remains in the forward position.



Fig. 1-40



Fig. 1-41

- Press **YES**.

1.7.3 *Washing the lines*

The analyzer will take approximately one minute to flush all lines with water. Upon completion, you will receive the prompt [**WITHDRAW WATER PRESS YES**].

- Remove the line with the blue mark from the water and place it on a cloth.
- Press **YES**. During this cycle, all lines are purged of water.
- Upon completion, [**ALL ELECTRODES REMOVED?**] will be displayed.



Fig. 1-42

1.7.4 *Removing the electrodes and the shutdown plug*

- Remove the shutdown plug from the Fluid Pack receptacle.
- Unplug the reference connector below the left side of the electrode holder.
- Move the lever on the left side forward and remove all electrodes, placing them on a soft cloth.
- Move the lever back and push the empty electrode holder into its back position. Unscrew the transport housing from the reference housing.
- Empty the reference housing and place all electrodes into their protective boxes.
- Press **YES**.

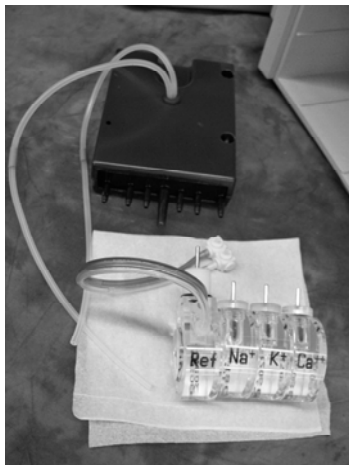


Fig. 1-41

1.7.5 *Installing the relief clamps and removing the pump windings*

The display will prompt [VALVE CLAMPS INSTALLED?].

- Install the red relief clamps making sure that they are snapped securely into position.

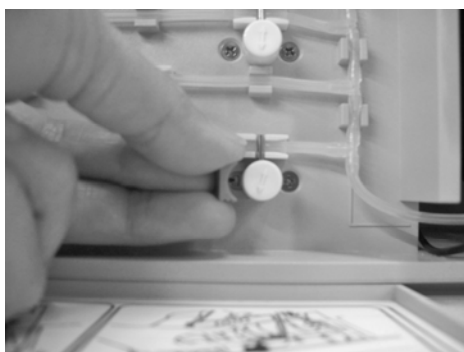


Fig. 1-44

- Press **YES**.



Never insert the solenoid relief clamps with the ISE Pack in place.

- The prompt [**PUMP TUBING RELIEVED?**] appears.
- Grasp the front winding close to the pump roller and gently pull it off the roller.
- Repeat the same procedure for the rear pump winding.



Fig. 1-45

- Press **YES**. The prompt [**RESET ALL DATA?**] will appear.
- By pressing YES, all QC values and statistics will be deleted and the sample number is reset to 0. QC and normal ranges as well as correlation factors, printer settings, date/time and ISE Pack volume are reset to default. Also, all service codes are deactivated. A deletion of all data is recommended in case the analyzer is operated by different personnel later on.
- Press **NO** to retain all data.

1.7.6 *Turning the analyzer "off"*

- The prompt [**SHUTDOWN EXECUTED POWER OFF**] will appear.
- Turn the analyzer off.
- Unplug the power cord from the receptacle.
- Close the front door.
- Clean all external surfaces of the analyzer as well as the areas accessible through the main door and sample door (see chapter 5: "Maintenance", section "Clean sample probe and fill port").
- Shutdown is complete.
- Use the original packaging when transporting the analyzer.

1.8 *Standby mode*

The InSight Electrolyte Analyzer is designed to calibrate automatically every four hours during normal operation. If sampling will be delayed for an extended period of time, such as evenings and weekends, you may place the analyzer into Standby mode to suspend automatic calibration.

TIP: *The analyzer can be programmed to automatically enter Standby mode. For instructions see chapter 6, section "Service codes".*

To enter standby mode:

- Press **NO** until [**OPERATOR SETTINGS?**] appears.
- Press **YES**, then **NO**, until [**ACTIVATE STANDBY?**] appears. Press **YES**.

The analyzer shows a Standby mode status on the display. [***STANDBY* YES → READY**]

To exit from Standby mode:

- Press **YES**, [**DEACTIVATE STANDBY?**] will be displayed.
- Press **YES** again.

Depending on the duration of the standby mode, an automatic calibration may be performed.

After that, the analyzer returns to [**H-READY**].

1.9 *Daily auto calibration*

When the instrument is in Standby, it can be set to do an automatic daily calibration at any time of the day. This allows the instrument to be ready when the operator requires it at a specific time each day.

To enable this function and to set the time for Auto Cal:

- Enter [**INSTRUMENT SETTING**] menu.
- Select [**AUTO CAL SETTINGS?**] by pressing **YES**.
- To set or change the time of the automatic calibration, press **YES** when [**CHANGE HR?**] is displayed.
- The time can be adjusted by selecting **NO** to scroll through the numbers.
- Then press **YES** when the correct number is displayed.

To disable this function:

- Enter [**INSTRUMENT SETTING**] menu.
- Select [**AUTO CAL SETTINGS?**] by pressing **YES**.
- At [**CHANGE HR?**], press **NO**.
- At [**DISABLE FIRST AUTOCAL**], press **YES** which will disable [**DAILY AUTO CAL**].

Auto Cal will occur when instrument is in Standby Mode.

2

2 Specifications

2.1	Measurement parameters	2-1
2.2	Performance parameters	2-1
2.2.1	Reproducibility	2-1
2.2.2	Linearity	2-2
2.2.3	Correlation Studies	2-2
2.3	Interferences	2-3
2.4	Limitations	2-3
2.5	Relationship of ionized calcium to total calcium	2-3
2.6	Bibliography	2-4
2.7	Sample throughput	2-5
2.8	Sample volumes	2-5
2.9	Sample types	2-5
2.10	Calibrations	2-5
2.11	Environmental parameters	2-6
	Temperature / humidity / stability	2-6
	Instrument	2-6
	Electrodes	2-6
2.12	Reagents	2-7
	ISE Pack	2-7
	Urine Diluent	2-9
	QC Material	2-9
2.13	Product data	2-10
2.13.1	Electrical data	2-10
2.13.2	Classification	2-10
2.13.3	Dimensions	2-10
2.14	Printer	2-11
2.15	Display	2-11

2 Specifications

2.1 Measurement parameters

	Matrix	Specified range
Na ⁺	B/P/S/Q	40 - 200 mmol/L
	U	3 - 300 mmol/L
K ⁺	B/P/S/Q	1.7 - 15 mmol/L
	U	5 - 120 mmol/L
Cl ⁻	B/P/S/Q	50 - 200 mmol/L
	U	15 - 300 mmol/L
iCa ²⁺	B/P/S/Q	0.3 - 5 mmol/L
Li ⁺	B/P/S/Q	0.2 - 5.5 mmol/L

B = Whole blood
Q = Aqueous QC

P = Plasma
U = Urine

S = Serum

Display Resolution:	Na ⁺ : 1 mmol/L	or	0.1 mmol/L
	K ⁺ : 0.1 mmol/L	or	0.01 mmol/L
	Cl ⁻ : 1 mmol/L	or	0.1 mmol/L
	iCa ²⁺ : 0.01 mmol/L	or	0.001 mmol/L
	Li ⁺ : 0.01 mmol/L	or	0.001 mmol/L

2.2 Performance parameters

2.2.1 Reproducibility

Typical Within-Run Precision is determined from 20 samples run between calibrations.

Reproducibility:
Within Run (n=20)

BLOOD/SERUM/PLASMA

Na⁺	C.V. ≤ 1% (120 – 160 mmol/L)
K⁺	C.V. ≤ 2% (2.5 – 6 mmol/L)
Cl⁻	C.V. ≤ 2% (85 – 130 mmol/L)
iCa	SD ≤ 0.02 mmol/L (0.8 – 1.5 mmol/L)
Li⁺	SD ≤ 0.02 mmol/L (0.4 – 1.3 mmol/L)

Reproducibility:
Within Run (n=20)

URINE

Na⁺	C.V. ≤ 5% (100 – 250 mmol/L)
K⁺	C.V. ≤ 5% (10 – 60 mmol/L)
Cl⁻	C.V. ≤ 5% (100 – 250 mmol/L)
(Calcium and Lithium are not typically measured in urine samples)	

2.2.2 Linearity

Whole blood, Plasma, Serum and Urine are linear across the claimed performance range. A minimum of 5 levels were tested for each type of sample. Dilutions were made from starting stock solutions and regression analysis done. Correlation coefficients were all greater than 0.99. Slopes were in the range of 0.97 to 1.02.

2.2.3 Correlation Studies

Studies were conducted comparing the InSight to the Roche 9180 with whole blood, plasma, serum and spot urine samples

Whole Blood in mmol/L

Parameter	Slope	Intercept	R ²	Range	n	Steyx
Sodium	0.988	1.04	0.9962	42.3 – 188.1	128	1.13
Potassium	0.972	0.223	0.9957	1.51 – 14.46	112	0.166
Chloride	1.011	-0.49	0.9882	54.0 – 191.4	123	1.76
Calcium	0.994	0.019	0.9885	0.304 – 4.514	124	0.076
Lithium	0.983	-0.049	0.9919	0.255 – 5.410	123	0.079

Plasma in mmol/L

Parameter	Slope	Intercept	R ²	Range	n	Steyx
Sodium	0.987	0.87	0.9977	43.5 – 194.4	105	1.06
Potassium	1.006	-0.014	0.9973	1.55 – 13.97	107	0.178
Chloride	1.022	-3.23	0.9879	55.2 – 193.2	115	2.58
Calcium	0.971	0.042	0.9915	0.363 – 4.386	123	0.059
Lithium	0.993	0.002	0.9878	0.298 – 5.071	115	0.092

Serum in mmol/L

Parameter	Slope	Intercept	R ²	Range	n	Steyx
Sodium	0.99	0.03	0.9964	44.6 – 196.9	122	1.37
Potassium	0.981	0.085	0.9979	1.58 – 14.64	127	0.118
Chloride	1.021	-2.57	0.9687	50.7 – 185.7	125	3.79
Calcium	0.970	0.036	0.9842	0.655 – 4.309	120	0.082
Lithium	1.004	0.092	0.9876	0.213 – 5.149	131	0.122

Urine (Spot) in mmol/L

Parameter	Slope	Intercept	R ²	Range	n	Steyx
Sodium	0.983	-2.57	0.9843	8– 296	118	8.42
Potassium	0.965	0.34	0.9934	5.3 – 118.2	108	2.22
Chloride	0.976	-1.86	0.9807	15 – 272	118	8.99

2.3 Interferences

Negatively charged ions are known to interfere with the chloride electrode causing a positive bias. Salicylate is a common interferent and in its clinical range causes a positive bias of approximately 1 mmol/L which is clinically insignificant. Other ions such as Thiocyanate and Bromide can also cause a positive bias in the chloride ion.

Interferent	Interferent	Concentration Effect
Bromide	1 mmol/L	increases chloride results by 13 mmol/L
Thiocyanate	2 mmol/L	increases chloride results by 2 mmol/L
Albumin	20 g/L	decreases chloride results by 4 mmol/L
Bicarbonate	10 mmol/L	decreases chloride results by 1 mmol/L



Use only lithium-free sampling containers for the determination of lithium measurement values! If sample containers are used which contain lithium as anticoagulant, this may lead to incorrect patient measurements, which may result in incorrect clinical decisions, possibly endangering the patient's health.

2.4 Limitations

A number of substances have been reported to cause physiological changes in blood, serum and plasma analyte concentrations. Medications and endogenous substances can affect results and clinicians must evaluate results based on the patient's entire clinical situation.

A comprehensive discussion concerning these and other interfering substances, their blood, serum or plasma concentrations, and their possible physiological involvement is outside the scope of this *Operators Manual*.

Opening and closing the fist with a tourniquet in place can result in an increase in potassium levels. It is recommended that the blood sample is obtained without a tourniquet, or that the tourniquet be released after the needle has entered the vein.

Since the concentration of potassium inside erythrocytes is much greater than that in extra cellular fluid, hemolysis should be avoided. Serum and plasma should be separated from the cells as soon as possible after collection.

The lithium electrode response is dependent on the actual sodium concentration of the sample. Sodium concentrations between 95 and 180 mmol/L (89.6 -169.5 mmol/L for blood/serum) are required for lithium values to be reported.

2.5 Relationship of ionized calcium to total calcium

The ratio of ionized calcium to total calcium in a healthy population is around 0.50 or 50%¹. These relationships may be altered when using citrate in blood, or when the acid base metabolism is disturbed.

¹ Tietz, Norbert W., Ed., Clinical Guide to laboratory Tests, 2nd Ed., (Philadelphia: W.B.Saunders, Co., 1990) p.98-99, 118-119, 456-459, 510-511, 720-721.

2.6 Bibliography

Bishop ML, Duben-Engelkirk JL, Fody EP. Clinical Chemistry Principles Procedures Correlations, 2nd Ed., (Philadelphia: J.B.Lippincott Co.),1992,p.281.

Burritt MF, Pierides AM, Offord KP: Comparative studies of total and ionized serum calcium values in normal subjects and in patients with renal disorders. Mayo Clinic Proc. 55:606, 1980.

Burtis C, Ashwood E (Eds.), Tietz Textbook of Clinical Chemistry, 2nd Ed., (Philadelphia: W.B. Saunders, Co.,1994) pp.1354-1360,2180-2206.

Calbreath, Donald F., Clinical Chemistry A Fundamental Textbook, (Philadelphia: W.B. Saunders Co., 1992) pp.371, 376, 390-395.

National Committee for Clinical Laboratory Standards. Protection of Laboratory Workers from Occupationally Acquired Infections, Second Edition; Approved Guidelines; NCCLS Document M29-A2, (2001).

National Committee for Clinical Laboratory Standards. Additives for Blood Collection Devices: Heparin; Tentative Standard; NCCLS Document H24-T, (1988).

National Committee for Clinical Laboratory Standards. Evaluation of Precision Performance of Clinical Chemistry Devices, Second Edition; Tentative Guideline. NCCLS Document EP5-T2, (1992).

Tietz, Norbert W.,Ed., Textbook of Clinical Chemistry, 2nd Ed., (Philadelphia: W.B. Saunders, Co.,1986), pp.1816, 1837, 1840-1842, 1845.

Toffaletti J, Gitelman JH, Savory J: Separation and quantification of serum constituents associated with calcium by gel filtration. Clin Chem 22: 1968-72, 1976.

2.7 *Sample throughput*

45 per hour with printout

60 per hour without printout

2.8 *Sample volumes*

95 µL typical

2.9 *Sample types*

Whole blood, serum, plasma, urine and Q C material

2.10 *Calibrations*

1-point calibration after each sample

2-point or 3-point calibration every 4 hours

2.11 *Environmental parameters*

Temperature / humidity / stability

Instrument

Operating conditions:

Ambient temperature	15 - 32 °C
Relative humidity	< 85% (not condensing)

Storage and transportation conditions:

Temperature	-20 - 40 °C
Humidity	< 85% (not condensing)

Electrodes

Operating conditions:

Operating temperature	15 - 33 °C
Relative humidity	20 - 95%, if $T \geq 15$ to ≤ 31 °C 20 - 90%, if $T > 31$ to ≤ 33 °C

Storage conditions in original package:

Temperature.....	18 – 25 °C (dry)
Humidity	15 - 85% (not condensing)

2.12 Reagents

ISE Pack (PN AV-BP5186D)

Contains the following reagents:

Standard A

For calibration of sodium, potassium, chloride, ionized calcium and lithium in the InSight Electrolyte Analyzer.

Contents:	350 mL										
Active ingredients:	<table> <tr> <td>Na⁺</td><td>150 mmol/L</td></tr> <tr> <td>K⁺</td><td>5.0 mmol/L</td></tr> <tr> <td>Cl⁻</td><td>115 mmol/L</td></tr> <tr> <td>Ca²⁺</td><td>0.9 mmol/L</td></tr> <tr> <td>Li⁺</td><td>0.3 mmol/L</td></tr> </table>	Na ⁺	150 mmol/L	K ⁺	5.0 mmol/L	Cl ⁻	115 mmol/L	Ca ²⁺	0.9 mmol/L	Li ⁺	0.3 mmol/L
Na ⁺	150 mmol/L										
K ⁺	5.0 mmol/L										
Cl ⁻	115 mmol/L										
Ca ²⁺	0.9 mmol/L										
Li ⁺	0.3 mmol/L										
Additives:	Germicides										
Storage temperature:	18 - 25 °C										
Stability:	Expiration date and lot number are printed on each container label										
On-board stability:	14 weeks										

Standard B

For calibration of sodium, potassium, chloride, ionized calcium and lithium in the InSight Electrolyte Analyzer.

Contents:	85 mL										
Active ingredients:	<table> <tr> <td>Na⁺</td><td>100 mmol/L</td></tr> <tr> <td>K⁺</td><td>1.8 mmol/L</td></tr> <tr> <td>Cl⁻</td><td>72 mmol/L</td></tr> <tr> <td>Ca²⁺</td><td>1.5 mmol/L</td></tr> <tr> <td>Li⁺</td><td>0.3 mmol/L</td></tr> </table>	Na ⁺	100 mmol/L	K ⁺	1.8 mmol/L	Cl ⁻	72 mmol/L	Ca ²⁺	1.5 mmol/L	Li ⁺	0.3 mmol/L
Na ⁺	100 mmol/L										
K ⁺	1.8 mmol/L										
Cl ⁻	72 mmol/L										
Ca ²⁺	1.5 mmol/L										
Li ⁺	0.3 mmol/L										
Additives:	Germicides										
Storage temperature:	18 - 25 °C										
Stability:	Expiration date and lot number are printed on each container label										
On-board stability:	14 weeks										

Standard C

For calibration of sodium, potassium, ionized calcium and lithium in the InSight Electrolyte Analyzer.

Contents:	85 mL	
Active ingredients:	Na ⁺	150 mmol/L
	K ⁺	5.0 mmol/L
	Cl ⁻	115 mmol/L
	Ca ²⁺	0.9 mmol/L
	Li ⁺	1.4 mmol/L
Additives:	Germicides	
Storage temperature:	18 - 25 °C	
Stability:	Expiration date and lot number are printed on each container label	
On-board stability:		
	14 weeks	

Reference Solution

A salt bridge for calibration and measurement in the InSight Electrolyte Analyzer.

Contents:	85 mL
Active ingredients:	Potassium, chloride 1.2 mmol/L
Additives:	Germicides
Storage temperature:	18 - 25 °C
Stability:	Expiration date and lot number are printed on each container label
On-board stability:	14 weeks



*On-board stability refers to the ISE Pack installed in the analyzer.
The ISE Pack has to be exchanged at the latest after 14 weeks.*



*Dispose of the ISE Pack according to local regulations for hazardous waste material.
A waste container is provided with the ISE Pack which holds all waste fluids used, therefore, all sample types, calibrators, cleaning solution, and conditioning solution. This must be handled with appropriate care due to its infectious potential. Avoid skin contact or ingestion.*

Urine Diluent (PN AV-BP0344D)

Use as a diluent for the measurement of urine samples in the InSight Electrolyte Analyzer.

Contents:	500 ml of solution
Active ingredients:	120 mmol/L Sodium Chloride
Additives:	Germicide
Storage temperature:	18-25°C
Stability:	Expiration date and lot number are printed on each container label

QC Material (PN DD-92001, DD-92002, and DD-92003, or DD-92123)

Mission Controls	Single Level 1, 2, and 3 (DD-92001, DD-92002, and DD-92003) Tri-level, 10 ampules of each level (DD-92123)
Contents:	30 ampules total
Storage Temperature:	18 - 25 °C

2.13 *Product data*

2.13.1 *Electrical data*

Mains voltage range 100 to 240 VAC

Frequency 50/60 Hz

Required power 50 W

2.13.2 *Classification*

Safety Category I

Overvoltage categoryII

Contamination level..... 2

2.13.3 *Dimensions*

Height..... 33.5 cm

Width..... 31.5 cm

Depth..... 29.5cm

Weight < 6 kg

2.14 Printer

Type.....thermal printer
Resolution.....16 characters / line
Printing speed.....4 mm / sec
Paper width.....38 mm
Paper length.....about 30 m

2.15 Display

Type..... monochrome LCD
Resolution2 lines, 16 characters per line

3

3 *Measurement*

3.1	Preanalytic	3-1
3.1.1	Sample collection.....	3-1
	Safety.....	3-1
	Sample requirements.....	3-1
	Acceptable anticoagulants	3-1
	Sample collection containers	3-2
3.1.2	Sample handling.....	3-2
	Whole blood	3-2
	Serum	3-2
	Plasma	3-2
	Aqueous solutions.....	3-3
3.2	Limitations of clinical analysis	3-3
3.2.1	General.....	3-3
3.2.2	Electrolytes	3-3
3.3	Measuring procedure.....	3-4
3.3.1	Sample measurement.....	3-4
3.3.2	Direct ISE	3-6
3.3.4	Urine samples	3-7
3.4	Normal ranges	3-8
3.5	Correlation factors	3-9
3.5.1	Determination of the correlation factor.....	3-10
3.6	Selecting Vet Mode.....	3-10

3 Measurement

3.1 Preanalytics

3.1.1 Sample collection

Safety



Follow the usual applicable safety precautions when drawing blood samples. When handling blood samples, there always exists the danger of transmission of HIV, hepatitis B and C viruses or other pathogens transmissible by blood. Employ suitable blood sampling techniques in order to reduce risk to personnel. Always wear protective gloves and suitable protective clothing.

Please refer to NCCLS document M29-A2, "Protection of Laboratory Workers from Occupationally Acquired Infections", Approved Guidelines - Second Edition 2001, for further information on safe handling of these specimens.

Sample requirements

Only qualified personnel may perform the collection of blood needed for analytical purposes.



The puncture site may never be squeezed! Mixing the blood sample with tissue fluid may lead to the premature onset of clotting despite sufficient heparinization of the sample collection containers! Incorrect sample collection or the use of an unsuitable sample collection container may lead to errors and discrepancies in the measurement values.

See, for example, NCCLS document H11-A3, "Procedures for the collection of arterial blood specimens - Third Edition", Approved Standard, 1999, for detailed information about blood sampling, storage, and handling.

Hemolyzed samples and icteric samples should not be used since their interferent effect on the InSight has not been tested.

Acceptable anticoagulants

The only clot inhibitors that may be used for analyses in the InSight Electrolyte Analyzer are heparin salts. Other clot inhibitors, such as EDTA, citrate, oxalate, fluoride, and ammonium-based materials have significant influence on parameters and may not be used for this reason.



Use only lithium-free sampling containers for the determination of lithium measurement values! If sample containers are used which contain lithium as anticoagulant, this may lead to incorrect patient measurements, which may result in incorrect clinical decisions, possibly endangering the patient's health.

Sample collection containers

The InSight Electrolyte Analyzer will accept samples directly from syringes, collection tubes, and sample cups.

3.1.2 *Sample handling*

For ionized calcium values, anaerobic conditions must be followed for all sample types. Contact with ambient air will cause a loss of CO₂ in the sample and the subsequent rise in pH will cause a reduction in ionized calcium.

Hemolyzed samples and icteric samples should not be used since their interferent effect on the InSight has not been tested.

Whole blood

Withdraw whole blood samples using heparinized syringes, capillaries, or the microasampler.

Analyze the samples as soon as possible after sampling. Remove air bubbles from the sample collection container immediately after the sampling procedure.

Serum

After the appearance of spontaneous clotting, process the sample in a centrifuge to separate the cellular, solid components and the fibrin from the watery serum. Transfer the serum to a suitable sample container and seal.

If it is necessary to store the sample, close the sample container tightly and store between 4 - 8 °C. If a sample is cold, warm it to room temperature (15 - 33 °C) before analysis.

Plasma

Plasma samples are obtained by centrifuging heparinized whole blood, during which the cellular components of the blood are removed from plasma.

Complete the analysis as quickly as possible.

If it is necessary to store the sample, close the sample container tightly and store between 4 - 8 °C. If a sample is cold, warm it to room temperature (15 - 33 °C) before analysis.

Plasma samples older than 1 hour must be re-centrifuged in order to remove fibrin clumps that may have formed.

Each laboratory should determine the acceptability of its own blood collection syringes, and tubes and the serum or plasma separation products. Variations in these products exist between manufacturers, and at times, from lot to lot.

Aqueous solutions

Aqueous samples, such as Standard A, have to be measured in the Standard Mode.



The Standard Mode ALWAYS reports direct ISE values and is not affected by correlation factors or the setting of QCF (see 'Service Codes').

3.2 Limitations of clinical analysis

The determined performance data can be influenced by known and unknown factors as described below.

3.2.1 General

The literature lists various substances which may negatively affect the measurement result of the blood and plasma/serum sample material. A detailed discussion of these phenomena can be found at different places in the technical literature. With respect to the InSight Electrolyte Analyzer, an attempt was made to identify or evaluate these possible influences. But since it is not possible to check all medication or substances, the user should be immediately informed with abnormal deviations of the measurement results as with every clinical analysis and evaluate the complete picture of the patient or perform expanded measurements in his own laboratory, if necessary.

3.2.2 Electrolytes

It is well-known, for example, that the potassium value of a patient can vary by up to 20% from the normal state, simply because of the presence of a pressure bandage. Hence, taking a blood sample while a pressure bandage is present should be avoided. In general, a local hemolysis caused by pressure should be avoided prior to taking a blood sample.

3.3 Measuring procedure



QC measurements must be performed in their entirety (i.e., all three QC levels must be measured). Omitting QC measurements or ignoring QC measurement result may lead to incorrect patient measurements, which may result in incorrect clinical decisions, possibly endangering the patient's health.

In order to ensure the quality of the measurement results, complete a quality control test on 3 levels (low, normal, high) after each electrode change and after startup of the instrument. It is recommended that this be done at least once daily or more often in accordance with local regulations.

3.3.1 Sample measurement

The InSight Electrolyte Analyzer provides fast and easy operation. Whenever [**H-READY**] appears, the analyzer is ready to accomplish sample measurements.



It is very important that the main door is closed during sampling to provide shielding from sources of electromagnetic interference.



Urine samples require dilution, and must be analyzed in the urine mode. Instructions for analyzing urine samples: see "Urine samples".



*The InSight provides the ability to enter the patient name and ID associated with the sample being tested. To enable this feature, press **YES** at [**OPERATOR SETTINGS?**]. Select **NO** until [**PATIENT DATA ENTRY SETTING?**] is displayed and then press **YES**. This enables patient data entry.*

*TIP: If **YES** is pressed, the InSight will ask for ID# and name if enabled. Opening the door will not allow the operator to enter a name or ID#. No name or ID# will be assigned to the sample.*

To take a sample measurement, follow procedure below.

- Open the sample door when [**H-READY**] is displayed or press **YES**. The prompt [**LOAD SAMPLE**] will be displayed and the pump will start to aspirate.



Fig. 3-1

- Hold the ampule under the probe until **[WIPE PROBE_SHUT SAMPLE DOOR]** is displayed.
- Use a lint-free tissue to wipe the probe. Then close the sample door when prompted.



Fig. 3-2



It is very important that the sample probe is carefully cleaned each time following a sample introduction.

The analyzer will display **[THANK YOU]** and a brief countdown will begin. At completion of analysis, the test results will be displayed and printed.

Na	K	Cl
143.9	4.93	103.5

Fig. 3-3

ISE ANALYZER
H-InSight
NA-K-Cl
12-JUN-11 14:58
HOSPITAL EAST (optional)
NAME: _____ (optional)
ID: _____

TEST: SERUM

TEST #: 72

Na=143.9 mmol/L
K = 4.93 mmol/L
Cl = 103.5 mmol/L

Fig. 3-4



Values that are higher or lower than the programmed normal range will be indicated by an arrow pointing up or down.



If the Na⁺ result is above 180 mmol/L (above 169.5 mmol/L for blood/serum) or below 95 mmol/L (below 89.6 mmol/L for blood/serum), the Li⁺ value cannot be derived.

If an additional sample report is desired or the automatic sample report is turned off, the results may be printed by following these steps:

- Press **NO**.
- The prompt appears. [**PRINT REPORTS?**]
- Enter **YES**.
- The prompt is displayed. [**PRINT LAST SAMPLE?**]
- Enter **YES**.
- The measurement results will be displayed, and the report will print.
- The prompt appears. [**PRINT BY SAMPLE ID?**]
- Enter **YES**.
- The prompt is displayed. [**ENTER ID:**]
- Enter **NO** to scroll through the numbers and select **YES** when the correct number is shown or use keyboard or barcode.
- The InSight will begin printing.
- The prompt [**FIND MORE?**] is displayed which allows the user to find more results for the same sample ID performed at another day or time. Press **YES** to continue or **NO** to move to the next menu option.
- If **NO** is selected, the prompt [**PRINT BY NAME?**] is shown.
- Enter **YES**.
- Enter **NO** to scroll through the letters and select **YES** when the name is displayed.
- The InSight will begin printing.
- The prompt [**FIND MORE?**] is displayed which allows the user to print more results from another test for the same patient name. Press **YES** to continue or **NO** to move to the next menu option.
- The prompt is displayed. [**PRINT BY TEST?**]
- Enter **YES**
- At the prompt, [**ENTER TEST #:**] enter **NO** to scroll through the numbers and select **YES** when the correct number is shown.
- The InSight will begin printing.
- The prompt [**FIND MORE?**] is displayed which allows the user to print results with the same test number. Press **YES** to continue or **NO** to move to the next menu option.
- The prompt appears. [**PRINT BY DATE?**]. This will print all results for a specific day.
- [**DAILY SUMMARY**] will print the last day's results.
- [**WEEKLY SUMMARY?**] prints the results from the previous 5 days.
- [**PRINT ALL?**] will print all data stored. This function can be interrupted by selecting **NO**.



Stored results can also be deleted.

3.3.2 Direct ISE

- Press **NO** until [**QC/STD/URINE SAMPLE?**] appears; press **YES** and then **NO**, until [**STANDARD SAMPLE?**] is displayed. Press **YES** to confirm.



The Standard Mode ALWAYS reports direct ISE values and is not affected by correlation factors or the setting of QCF (see 'Service Codes').

3.3.3 Dialysate Analysis

- Press **NO** until [QC/STD/DIALYSATE URINE SAMPLE?] appears; press **YES** and then **NO**, until [BICARBONATE?] or [ACETATE?] is displayed. Press **YES** to perform the desired test(s).

NOTE: By default dialysate mode is **OFF**. Please contact your customer support to activate the dialysate mode.

3.3.4 Urine samples

Before measuring urine, accurately dilute the sample with Urine Diluent in the ratio of 1 part urine to 2 parts diluent (e.g., 1 mL urine and 2 mL urine diluent). Thoroughly mix the sample and analyze in the urine mode.

As soon as [H-READY] is displayed, the analyzer is ready for measurements. Urine samples, diluted with urine diluent, are analyzed in the urine mode. To enter this mode:

- Press **NO**, until [QC/STD/ URINE SAMPLE?] is displayed. Enter **YES**.
- Press **NO**, until [URINE SAMPLE?] is displayed. Enter **YES** and follow the instructions.
- Upon completion of measurement, the analyzer will display and print out the results.

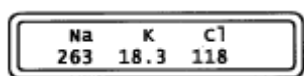


Fig. 3-4

ISE ANALYZER
H-InSight
Na-K-Cl
12-JUN-11 15:06

HOSPITAL EAST

ID: _____

TEST: URINE

TEST#: 29

Na=263 mmol/L
K =18.3 mmol/L
Cl =118 mmol/L

Fig. 3-5

ISE ANALYZER
H-InSight
Na-K-Li
12-JUN-11 15:06

HOSPITAL EAST

ID: _____

TEST: URINE

TEST#: 30

Na=142 mmol/L
K = ↑ ↑ ↑ mmol/L

Fig. 3-6



Ca^{2+} and Li^{+} are not measured in urine.

If the result of the urine sample is given with ↑↑↑ the K^{+} value of the sample is higher than 45 mmol/L, and outside the measurement range of the analyzer. The measurement must be repeated using the following procedure:

- Record the Na^{+} value (as well as Cl^{-} , if activated) of the first urine measurement.
- Dilute the diluted urine (already diluted 1:2 with urine diluent) with distilled water in the ratio of 1:2 (e.g., 1 mL of diluted urine and 2 mL distilled water).
- Thoroughly mix the sample.
- Run a second urine measurement with the twice-diluted urine sample.
- Ignore the Na^{+} value (and Cl^{-} , if activated).
- Multiply the K^{+} value by 3 and record.

3.4 Normal ranges

The instrument has preset normal ranges for whole blood, plasma and serum samples which are:

Na ⁺	136 - 145 mmol/L
K ⁺	3.5 - 5.1 mmol/L
Cl ⁻	97 - 111 mmol/L
iCa ²⁺	1.0 - 1.30 mmol/L
Li ⁺	0.6 - 1.2 mmol/L

The normal ranges can be adapted to the respective laboratory-specific requirements.

² Values that are higher or lower than the programmed normal range will be indicated by an arrow pointing up or down.

The normal ranges can be changed through the programming menu. Follow the instructions for [INSTRUMENT SETTINGS?] in chapter 4.2 and enter 3-digit password.

- Press **NO** until [SETUP NORMAL RANGES?] is displayed. Press **YES**.
- The analyzer will display the normal ranges of the current parameter configuration:
Na LOW=xxx
Na HIGH=xxx ok?
 Normal ranges can be entered by pressing **NO**
- Press **NO** until the desired number is displayed, and press **YES** to advance to the next position. Repeat this for all numbers of the normal range.
- After entering the normal ranges, the new values will be displayed. Verify correctness of the ranges:
Na LOW=xxx
Na HIGH=xxx ok?

TIP: If an incorrect number was entered inadvertently, press **NO**, and the analyzer allows for retyping the normal range.

- Press **YES** and the display will advance to the next parameter.

Repeat the above procedure for all parameters. After all activated parameters have been programmed, the display prompts [SETUP FACTORS FOR CORRELATION?]. This allows programming of the correlation factors.

The expected normal urine values are given in the table below². It should be used only as a guide. Each laboratory or testing site should establish its own range of normal values, taking into account factors such as age, sex, diet, and other determinants of electrolyte levels.

Urine Expected Values

Na ⁺	40 - 220 mmol/day
K ⁺	25 - 120 mmol/day
Cl ⁻	110 - 250 mmol/day

² Burtis C, Ashwood E (Eds.), Tietz Textbook of Clinical Chemistry, 2nd Ed., (Philadelphia: W.B. Saunders, Co.,1994)

3.5 Correlation factors

The correlation factor allows for the correlation of InSight measurement results to other electrolyte analyzers. Activated correlation values are taken into consideration when sampling whole blood, serum, plasma and QC samples. They are not used for standard samples analyzed in the [QC/STD/URINE SAMPLE] mode. A separate set of correlation factors is available for urine samples.



In case the QC samples were switched to report direct ISE values (Code QCF, see 'Service Codes'), the values are NOT affected by correlation factors.



The measurement range as well as normal and QC ranges are ALWAYS checked against the DISPLAYED values. Therefore, you may have to adjust the QC and normal ranges to your correlation factors.

The correlation factors can be changed through the programming menu. Follow the instructions for [INSTRUMENT SETTINGS?] in chapter 4.2 and enter the password.

- Press **NO**, until [SETUP FACTORS FOR CORRELATION?].
- Enter **YES**, the analyzer prompts [RESET CORREL FACTORS?].
- Entering **YES** deletes all current correlation factors. The analyzer will then advance to the other correlation programming functions.
Entering **NO** shows the prompt [ENTER/CHECK CORREL. FCTR?] and allows to input or to verify correlation factors for each parameter.
Entering **YES** to the prompt [ENTER/CHECK CORREL. FCTR] will allow to change the correlation factors or to verify correlation factors already programmed.

The current values will be displayed, such as:

Na(b) = +00.0

Na(m) = 1.000ok?

- If the Na⁺ intercept (b) and the slope (m) are correct, press **YES**.
- To change these values, press **NO**.
The intercept and slope values may be adjusted by pressing **NO** to change the number, and by pressing **YES** to accept the number.
The programmed factors are automatically applied to blood/serum and QC samples.

Follow the same procedure for the intercept (b) and slope (m) for K⁺, Cl⁻, iCa²⁺ and Li⁺.



Only activated parameters can be programmed.

A separate set of correlation factors is available for urine samples.

- At the prompt [ENTER URINE CORREL. FACTORS?], press **YES**.
- Program the factors as described above.

Note that there are no urine correlation factors available for iCa²⁺ and Li⁺, since these parameters are not measured in urine mode.



Correlation factors can be printed from the **[PRINT REPORTS?]** menu. To do this, press **NO** until **[PRINT CORRELATION FACTORS?]** is displayed. Then press **YES**.

3.5.1 Determination of the correlation factors

If the analyzer to correlate with does not compare to flame photometer values, the correlation factors need to be determined. This may be done using one of the following two methods:

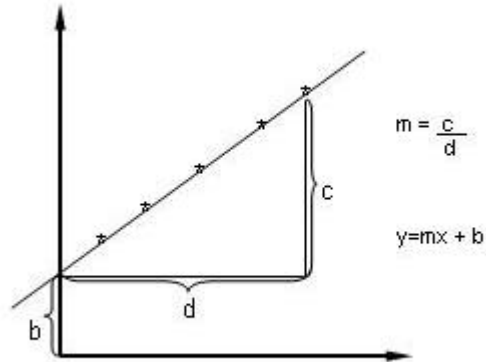


Fig. 3-7

a) Worksheet for the correlation factors

- 1) Measure MISSION CONTROLS Level 1 and Level 3, three times each on the InSight Electrolyte Analyzer and the reference analyzer.
- 2) Fill out a Correlation Factor Worksheet for each parameter. Follow the arrows and calculations to determine the intercept (b) and slope (m). (See appendix for blank worksheets).

b) Sample Analysis Regression function

- 1) Analyze at least 20 lipid-free serum samples or samples with normal lipids on the InSight Electrolyte Analyzer and on the reference analyzer. Select samples with different concentrations in order to provide determination points from the lowest point to the highest point of the measurement range.
- 2) Use a calculator with a linear regression function to determine the intercept (b) and slope (m).

3.6 Selecting Vet Mode

Vet Mode can be selected from the Operator Settings menu. Contact your local distributor to obtain the password. See Section 8.7, 8.8 for flow charts, instructions and normal ranges.

Once changed over to Vet Mode, verify that the display is **[V-READY]**.

4

4 *Quality Control*

4.1 General QC concept	4-1
4.2 Material setup.....	4-2
4.3 Performing a QC measurement	4-4
4.4 Printing a QC report.....	4-5

4 Quality Control



QC measurements must be performed in their entirety (i.e., all three QC levels must be measured). Omitting QC measurements or ignoring QC measurement results may lead to incorrect patient measurements, which may result in incorrect clinical decisions, possibly endangering the patient's health.

4.1 General QC concept

In order to ensure the quality of the measurement results, complete quality control tests on 3 levels (low, normal, high) after each electrode change and startup of the instrument. A quality control program for electrolytes includes the analysis of sample materials with known ranges of expected values and the comparison of these values with analyzer results.

The following control material is recommended:

- MISSION CONTROLS

The target values listed in the package insert sheet should be taken as 2 SD values (SD = standard deviation).

The QC measurement results within the target value range $\pm 2SD$ are acceptable.

If QC measurement results fall outside the target value range $\pm 3SD$, the parameter must not be used for further measurements!

QC measurement results that are greater than the target value $\pm 2SD$, but less than the target value $\pm 3SD$, should be investigated.

Possible cause of error:	Remedial action(resolution)
1. Low volume of ISE Pack	1. Replace ISE Pack
2. Inadequate reagent aspiration	2. a. Check for blocks in tubing or manifold on ISE Pack b. Initiate a calibration.
3. Electrode	3. a. Check fill solution level and for bubbles in electrode b. Check for proper flow of reference solution into reference electrode c. Initiate a calibration.
4. Pump Tubing may be stretched	4. a. Check if tubing loose b. Replace pump tubing c. Initiate a calibration.
5. High Temperature may affect Calcium results	5. Check temperature, See Section Check Temp steps

4.2 Material setup

The QC material must be defined prior to the QC measurement.

- Starting from the [**H-READY**] screen, press **NO**, until the prompt [**INSTRUMENT SETTINGS?**] appears, and select with **YES**.



For safety and security, the analyzer can only be programmed or have existing parameters changed by entering the correct password.

The analyzer prompts [**ENTER PASSWORD**]. The code "**DDI**" has to be entered as follows:

- Press **NO**, until the character D appears.
- Press **YES** and the cursor will advance to the second position.
- Press **NO**, until the character D appears.
- Press **YES** and the cursor will advance to the last position.
- Press **NO**, until I appears.

*TIP: If the desired character was missed, keep pressing the **NO** key until the correct character appears.*

If the code was entered correctly, press the **YES** key. The analyzer is now prepared for programming and will display [**SETUP CONTROL LEVEL 1 RANGES?**].

*TIP: If an incorrect code was entered inadvertently, the analyzer will display [**INVALID PASSWORD RETRY?**]. Enter **YES**, the [**ENTER PASSWORD**] prompt is displayed. To exit this menu, press **NO**.*

- To enter a new lot number, press **YES**.
- The analyzer prompts [**CURRENT LOT: xxxxx CHANGE LOT #?**].
- Enter **YES**, the analyzer shows [**PRINT OLD VALUES AND STATS?**].
- Enter **YES** to receive a printout that contains information including the mean, standard deviation (**SD**) and coefficient of variation (**CV**) of stored data.
- Enter **NO**, the analyzer shows [**NEW CONTROL LOT RESET DATA?**].



At confirming this prompt, all statistical data for current lot will be deleted.

- Enter **YES** to display [**ENTER CONTROL LOT #**]. Enter the new lot number by pressing the **NO** key until the desired number is displayed, and press **YES** to advance to the next position.

***TIP:** The first time QC lot number information is entered, the InSight Electrolyte Analyzer will display a default lot number. Thereafter, the current lot number will appear.*

*If the current lot information does not need to be changed, but the current programmed QC ranges need to be verified, press **NO**.*

TIP: *If an incorrect lot number was entered inadvertently, press **NO**, and the analyzer returns to [ENTER CONTROL LOT #].*

- The analyzer will now sequentially display high and low ranges for the electrolytes that correspond with the QC level and lot number, such as:

Na LOW = 040

Na HIGH = 205

- Press **YES** to display the next parameter range.
- After all activated parameters have been programmed, the display prompts **[ADDITIONAL PARAMETERS?]**. Select this option to program deactivated parameters.
- After each range has been displayed, QC level 1 programming will be completed.

To continue programming, repeat the above procedure for QC Level 2 and QC Level 3.

At the completion of QC Level 3 programming, the analyzer will prompt **[SETUP NORMAL RANGES?]**. This programming feature allows you to customize the normal ranges that the InSight Electrolyte Analyzer will use to flag abnormal patient measurement values on both the display and printed report (see chapter 8.1.2: "QC report").

4.3 Performing a QC measurement

In order to ensure the quality of the measurement results, perform a quality control test using all 3 levels (low, normal, high) after each electrode exchange and after startup of the instrument. Local regulations or your operating procedure may require quality control tests daily or more often.

TIP: Up to 35 QC measurements can be stored per level. The values can be stored and the statistics printed at any time.



MISSION CONTROLS should be stored at 18 - 25°C before use.



It is very important that the main door is closed during sampling, since it provides shielding from sources of electromagnetic interference.

Starting from the [H-READY] screen, press **NO** until [QC/STD/URINE SAMPLE?] appears.

- Press **YES**, [CONTROL LEVEL 1?] will be displayed. Press **YES**.

The prompt [LIFT DOOR LOAD SAMPLE] will be displayed.

- Take a Level 1 ampule from the MISSION CONTROLS box and mix it carefully.
- Gently tap the head of the ampule with the fingernail to remove any liquid.
- Carefully open the ampule by breaking off the top.



Protect fingers by using gloves or tissue while breaking ampule.

- Open the sample door: The prompt [LOAD SAMPLE] appears.

TIP: Introduce the QC material directly from the ampule!

Hold the ampule under the probe until [WIPE PROBE_SHUT SAMPLE DOOR] is displayed. Use a lint-free tissue to clean the probe, then close the sample door.



It is very important that the sample probe is carefully cleaned each time following a sample introduction.

The instrument will display [CONTROL LEVEL 1 TESTING...] and a countdown will begin, during which the QC will be analyzed. Upon completion, the results will be briefly displayed, such as:

Na	K	Cl
125.1	3.05	77.8



The InSight Electrolyte Analyzer "flags" values that are above or below the programmed target ranges by using an "up" or "down" arrow..

- The prompt [**SAVE QC VALUES?**] appears.
- To save the values in memory, press **YES**. To reject the values, press **NO**. If the values are rejected, the analyzer will return to the [**CONTROL LEVEL 1?**] display, allowing to repeat the QC Level 1 measurement by pressing **YES**, or to skip to QC Level 2 by pressing **NO**.



*If the value is outside the measurement range (↑↑↑↑,↓↓↓↓ or **ERR**), it will automatically be rejected.*

If the values have been saved, [**VALUES ACCEPTED!**] will be displayed shortly, followed by the prompt [**CONTROL LEVEL 2?**].

For further QC measurements (Level 2 and Level 3), follow the outlined instructions for Level 1, being sure to use Level 2 or Level 3 MISSION CONTROLS.

TIP: To discontinue QC measurement, press the **NO** key, until [**H-READY**] is displayed.

At completion of QC Level 3 measurement, the analyzer prompts [**CONTINUE QC/STD/URINE SAMPLE?**]. If measurements for all Levels have been completed, press **NO** and the analyzer returns to the [**H-READY**] display.

TIP: Responding **YES** to the [**CONTINUE QC/STD/URINE SAMPLE?**] prompt, the analyzer returns to [**CONTROL LEVEL 1?**].

4.4 Printing a QC report

The InSight Electrolyte Analyzer will store in memory the last 300 QC measurement values for each of the three QC levels.

To print the last QC value stored,

- Press **NO**, until [**PRINT REPORTS?**] is displayed, then press **YES**.
- [**PRINT QC & STATS**] press **YES**.
- At [**PRINT LAST CONTROL?**] press **YES**.
- Select the last stored results by pressing **YES** and **NO**.

To print statistical report,

- Press **NO**, until [**PRINT REPORTS?**] appears, then press **YES**.
- Press **NO** until [**PRINT QC AND STATS?**] appears. Press **YES**.
- Press **NO** until [**PRINT QC STATISTICS?**] appears. Press **YES**.
- Press **YES**, and select the QC statistics desired by level.



If correlation factors are changed or QC values switched to direct ISE, ALL values in memory are recalculated and printed in the new setting.

5

5 Maintenance

5.1	Decontamination	5-1
5.1.1	Decontamination Procedure.....	5-1
	Sample probe mechanism	5-1
	Surfaces of the instrument	5-1
5.1.2	Recomended disinfectants	5-2
5.2	Daily maintenance	5-2
5.3	Weekly maintenance.....	5-4
5.3.1	Clean sample probe and fill port.....	5-4
5.3.2	Cleaning analyzer surfaces	5-4
5.4	Monthly maintenance	5-5
5.4.1	Cleaning the reference electrode housing	5-5
5.5	Semi annual maintenance	5-8
5.5.1	Exchanging the pump tubing set.....	5-8
5.6	Annual maintenance	5-9
5.6.1	Exchanging main tubing harness	5-9
5.7	Unscheduled maintenance	5-10
5.7.1	Replacing electrodes.....	5-10
5.7.2	Checking reagent fluid level and changing the ISE Pack	5-11
5.7.3	Replacing sample probe and fill port.....	5-13
5.7.4	Replacing printer paper.....	5-14

5 Maintenance



After use, components of the InSight Electrolyte Analyzer, including tubing, fill port, ISE Pack etc., contain biological fluids and therefore represents a possible infectious risk.

Handle these components with care and according to regulations surrounding potentially infectious materials.

Avoid contact with skin! Always wear gloves! Danger of infection!

5.1 Decontamination

5.1.1 Decontamination Procedure

The purpose of this procedure is to minimize the risk of infections when replacing items that were in contact with blood.

Perform these decontamination procedures regularly.

The following decontamination procedure is recommended in addition to regulations specific to the laboratory.



Never use alcohol based disinfectants!

Never use disinfectant sprays.



IMPORTANT: Do not attempt to decontaminate any part of the instrument before shutting it down and unplugging it from the power source.

Before plugging the instrument back in and turning it on, always wait 15 minutes to allow the disinfectant to evaporate.

Regularly decontaminate the following parts of the instrument:

- Sample probe mechanism consisting of sample probe and fill port
- Surfaces of the instrument

Sample probe mechanism

See "Clean sample probe and fill port" on page 5-4.

Surfaces of the Instrument

See "Cleaning analyzer surfaces" on page 5-4.

5.1.2 Recommended disinfectant

Surfaces

Protein remover (PN AV-BP0521)

- Potential dangers
Due to the alkaline and oxidizing character of this preparation, we cannot rule out local irritation to the skin, eyes, and mucous membranes.
- First Aid measures
After inhalation: breathe fresh air.
After skin contact: wash with generous amounts of water, remove contaminated clothing.
After eye contact: rinse eyes with-generous amounts of water, contact an eye doctor
After drinking: drink large amounts of water, avoid vomiting, contact a doctor.

5.2 Daily maintenance

Prior to running the first sample of the day, the InSight Electrolyte Analyzer needs to undergo a simple cleaning and conditioning procedure that helps ensure the unit will perform properly. This procedure is called “maintenance” because it must be performed once each day the analyzer is used to conduct sampling.



In case cleaning and/or conditioning has not been performed within the last 24 hours, the analyzer will automatically print on each sample report [EXECUTE MAINTENANCE!].

TIP: *If fewer than 5 samples are analyzed each day, cleaning should be performed once a week instead of daily (see Weekly maintenance").*

- Press the **NO** key, until [MAINTENANCE?] is displayed, and **YES** to select.
- Select [CLEAN?] by pressing **YES**.
- At the prompt [LIFT DOOR LOAD SAMPLE], pour a small amount of Cleaning Solution A into a clean container. Lift the sample door and the pump will begin to aspirate.
- Hold the Cleaning Solution A under the probe, until analyzer beeps and the message [WIPE PROBE SHUT SAMPLE DOOR] is displayed, and use a lint-free tissue to remove the cleaning solution from the probe. Close the sample door.



Fig. 5-1



Fig. 5-2

- While the analyzer displays [**THANK YOU**] and a brief countdown, pour a small amount of conditioning solution into a clean container.
- At the prompt [**CONDITION?**], press **YES**.
- At the prompt [**LIFT DOOR LOAD SAMPLE**], lift the sample door and the pump will begin to aspirate.
- Hold the Electrolyte Conditioning Solution under the probe, until analyzer beeps and the message [**WIPE PROBE_SHUT SAMPLE DOOR**], use a lint-free tissue to remove the conditioning solution from the probe. Close the sample door.
- After the analyzer displayed [**THANK YOU**] and a brief countdown, press **NO** at the prompt [**CONTINUE MAINTENANCE?**]. An automatic calibration will be started.



It is very important that the main door is closed during calibration, since it provides shielding from sources of electromagnetic interference

5.3 Weekly maintenance

On a weekly basis, or whenever necessary, you should clean the sample fill port and sample probe as well as the exterior analyzer surfaces.

5.3.1 Clean sample probe and fill port

- Starting from the [H-READY] screen, perform the following actions:
Open the sample door.
- Clean the fill port, probe and surrounding area with a damp cotton swab.
- If the analyzer attempts to perform sample analysis, [SAMPLE NOT FOUND] will be briefly display, and it will return to the [H-READY] screen.
- The exterior surfaces should be wiped clean with a soft, damp cloth.



Never use strong or abrasive cleaners and no cleaners containing alcohol on the InSight Electrolyte Analyzer. Use a slightly damp cloth to avoid getting fluid inside the analyzer.



Fig. 5-3

5.3.2 Cleaning analyzer surfaces



*Never use alcohol based disinfectants!
Never use disinfectant sprays.*



IMPORTANT: Do not attempt to decontaminate any part of the instrument before shutting it down and unplugging it from the power source.

Before plugging the instrument back in and turning it on, always wait 15 minutes to allow the disinfectant to evaporate.

Regularly, decontaminate the surfaces of the analyzer including the front door with an undiluted disinfectant (Deproteinizer). Use swabs or sponge that have been soaked with Deproteinizer. The minimum deproteinizing disinfection contact time of 15 minutes is to be strictly followed.

Very dirty surfaces (e.g., contaminated with blood) should first be cleaned with swabs or a sponge that have been soaked with Deproteinizer disinfectant. If the grooves in the front door or the areas around the measuring chamber window require more cleaning, use a small commercial plastic brush to clean them. Remove all visible impurities. The deproteinizing disinfection procedure should be followed as described above.

5.4 Monthly maintenance

5.4.1 Cleaning the reference electrode housing

Monthly maintenance involves cleaning the reference electrode housing, and should be performed prior to daily maintenance. To complete this procedure, a small amount of deproteinizer is needed.

- From the [H-READY] screen, press **NO** to access [MAINTENANCE?].
- Press **YES**, the prompt [CLEAN?] appears.
- Do not press any keys.
- Open the front cover of the analyzer.
- Unplug the tubing of the reference electrode from the receptacle below the left side of the measuring chamber.

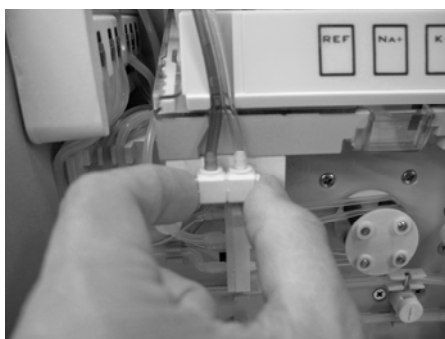


Fig. 5-4

- Slide the measuring chamber forward until it locks in the front position.
- Unclamp the left electrode holder by moving the clamp forward.



Fig. 5-5



Fig. 5-6

- Open the reference electrode assembly from the analyzer
- Unscrew the reference electrode from the reference electrode housing. Store the reference electrode in the transport housing filled with reference solution.

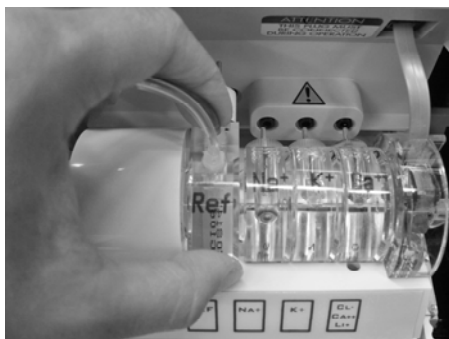


Fig. 5-7

TIP: The reference solution can be taken from the reference electrode housing.



It is very important that the reference electrode always be stored in reference solution and never allowed to become dry.

- Pour deproteinizer solution into a small container and submerge the reference housing into it, ensuring that no air bubbles remain in the housing.



Do not submerge the reference connector and tubing.

- After 15 minutes, remove the reference housing from the deproteinizer. Thoroughly rinse the housing with tap water and dry.
- Unscrew the transport housing from the reference electrode. Check that the O-ring on the electrode is properly seated. Rinse the transport housing with water and save for later use.
- Carefully screw the reference electrode into the reference electrode housing. Install the assembly into the left side of the measuring chamber.
- Close the clamp on the left electrode holder by lifting it upward until it locks in the back position. Ensure the electrodes are seated properly.

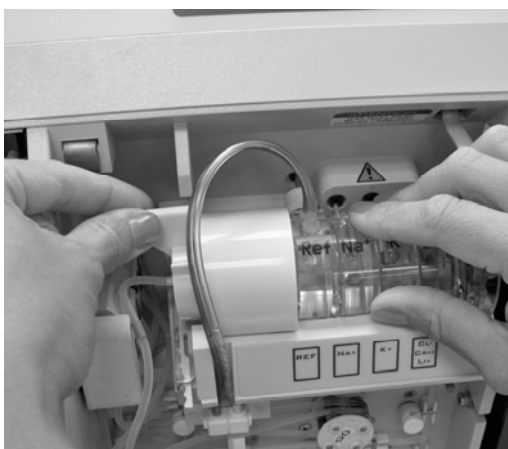


Fig. 5-8

- Plug the tubing of the reference electrode into the receptacle below the left side of the measuring chamber.
- Slide the measuring chamber back until it snaps into position, and close the front cover of the analyzer.

Tip: Measuring chamber may need to be tipped upwards to slide electrodes into this receptacle

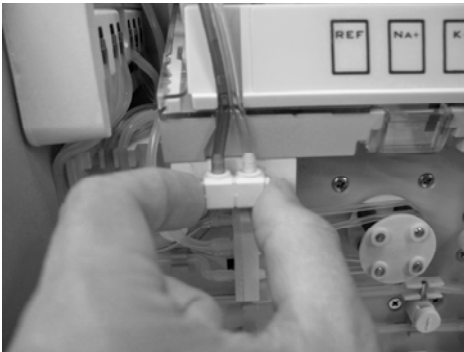


Fig. 5-9

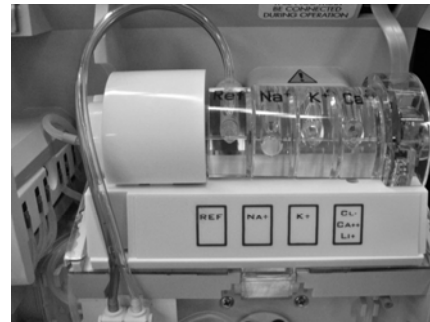


Fig. 5-10

- At this time, you may perform daily maintenance by pressing **YES**.
- If daily maintenance is not needed, press **NO** until [**PERFORM CALIBRATION?**] is displayed. Press **YES** and a complete calibration cycle will be performed.

5.5 *Semi annual maintenance*

5.5.1 *Exchanging the pump tubing set*

To make sure that the pump does not turn on during this procedure, press **NO** until [MAINTENANCE?] is displayed. Press **YES** and [CLEAN?] will appear. Then, do not press any keys.

When changing the pump tubing set, proceed as follows:

- Remove the front cover and slip the two pump windings from the pump rollers.
- Next, disconnect one tube at a time from the old pump winding plate and reconnect to the same place on the new pump winding plate.

Tip: Have the new pump tubing set ready to replace the old one



Fig. 5-11

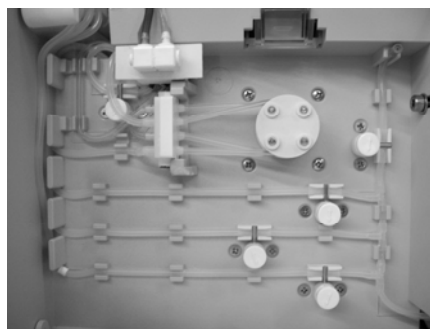


Fig. 5-12

Install the new pump winding plate and slip the new pump windings over the pump rollers, being careful not to cross the tubes.

Press **NO** until [PERFORM CALIBRATION?] is displayed. Press **YES** to initiate a calibration cycle.



Dispose of the pump tubing set according to local regulations for hazardous waste!

5.6 Annual maintenance

5.6.1 Exchanging main tubing harness

Once a year or as needed, the main tubing harness should be replaced.



Always wear gloves! Danger of infection!

- At the [**H-READY**] screen press **NO**, until [**SERVICE MENU?**] appears, and press **YES**.
- Press **NO**, until [**CHECK PINCH VALVES?**] appears. Press **YES**.
- Remove the valve caps, sliding them in the direction of the arrow.
- Then remove the pinch bars.
- Disconnect the reference electrode and pull out the white reference electrode receptacle.
- Disconnect the tubing with the green band from the left side electrode holder.
- Open the sample flap and pull out the fill port.
- Remove the ISE Pack and slide out the white ISE Pack receptacle. Disconnect the tubes from the pump winding plate and pull out the tubing harness.
- Re-install the new main harness in reverse order, use the tubing diagram in the front door as a guide for installation. Take special care to reconnect the tubes to the correct nipples of the pump winding.
- Activate each solenoid by pressing **YES**, then slide the pinch bar on and snap on the valve cap in the direction of the arrow.
- To activate the next solenoid, press **NO** twice. Notice the display text

TIP: *It may be necessary to pull on the metal tip of the valve in order to put the valve cap into position.*



Dispose of the main tubing harness according to local regulations (hazardous waste!).

TIP: *It is recommended to replace the sample probe and the fill port at the same time as the main tubing harness is replaced (see section 5.7.3: Replacing sample probe and fill port).*

- Press **NO** until [**PERFORM CALIBRATION?**] is displayed.
- Press **YES** to start a calibration.

5.6.2 Replacing Sample Probe

Once a year, the sample probe should be replaced.

- Disconnect tubing of Sample Probe from Sample Detector
- Open Sample Door
- Lift and remove Sample Probe from holder
- Place and secure new Sample Probe into holder
- Connect tubing of Sample Probe to the Sample Detector

5.7 *Unscheduled maintenance*

5.7.1 *Replacing electrodes*

- Slide the measuring chamber forward until it locks in the front position.
- Unclamp the left electrode holder by moving the clamp forward.

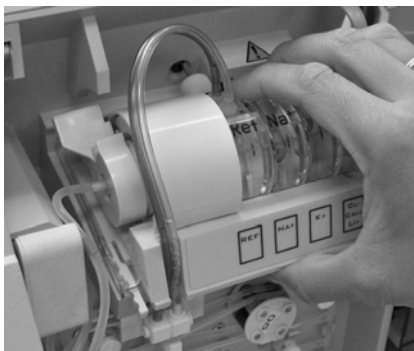


Fig. 5-13

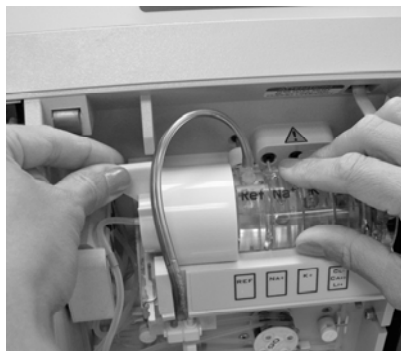


Fig. 5-14

- Remove the used electrode from the measuring chamber.



Fig. 5-15

- Remove the new electrode from its protective box and check for the presence of an o-ring in the left side of the electrode.
- Check for fill solution in the electrode according to the instructions enclosed with the electrode
- Install electrode in its labeled position in the measuring chamber. Note that the electrode has a lip on the bottom that rests on the flat edge of the measuring chamber to aid in proper positioning.

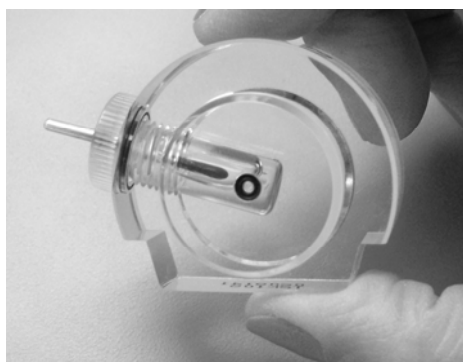


Fig. 5-16



Fig. 5-17

- Close the clamp on the left electrode holder by lifting it upward until it locks in the back position. Ensure that the electrodes are seated properly.
- Slide the measuring chamber back until it snaps into position.

Tip: The measuring chamber may need to be pushed upwards to position the electrode pins in the instrument receptacle



Fig. 5-18



Fig. 5-19

After installing a new electrode, the InSight Electrolyte Analyzer needs to undergo Daily Maintenance, Calibration and QC Measurement to verify the performance of the electrode.



Dispose of the electrodes according to local regulations hazardous waste!.



In order to ensure the quality of the measurement results, complete a quality control test on 3 levels (low, normal, high) after installation each electrode exchange

5.7.2 Checking reagent fluid level and changing the ISE Pack

The InSight Electrolyte Analyzer monitors the level of solutions in the ISE Pack and displays the amount remaining. To check the status of fluid remaining in the ISE Pack:

- In the [H-READY] screen, press the **NO** key, until [OPERATOR SETTINGS?] is displayed. Press **YES**.
- The prompt [VERIFY ISE PACK?] will be displayed. Press **YES**.
- The analyzer will display the amount of fluid remaining.



Always wear gloves! Danger of infection!

- To change the ISE Pack, just grasp the extended portion and slide it out.

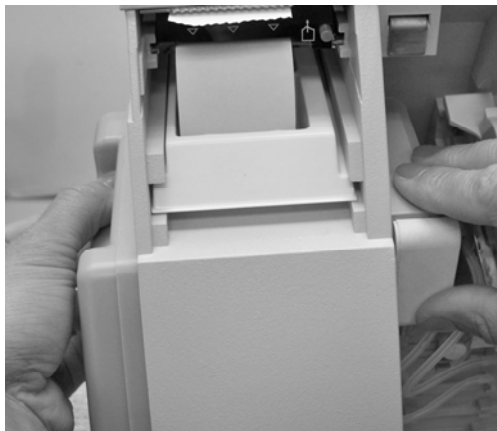


Fig. 5-20

As soon as the analyzer detects that the ISE Pack has been removed, the display prompts **[NO PACK INSTALLED]**. In addition, the printer will print the current status of the ISE Pack



Do not remove the ISE Pack during a calibration or measurement procedure.



Dispose of the ISE Pack according to local regulations for hazardous waste.

Prepare the new ISE Pack by carefully removing the protective strip. Save this cover to close the connectors prior to disposal of the used ISE Pack. Write the installation date on the label of the new ISE Pack.



Once the protective strip is removed, be sure to keep the ISE Pack upright to avoid spillage.

- Slide the new ISE Pack into position on the left side of the analyzer. The analyzer will prompt **[NEW PACK INSTALLED?]**.



Fig. 5-21



Fig. 5-22

- Press **YES** to indicate that a new ISE Pack is installed. Again, at the prompt [**ARE YOU SURE?**], press **YES**. The InSight Electrolyte Analyzer will automatically reset the ISE Pack counter to 100% and commence system calibration.
- Press **NO** if you reinstall the ISE Pack that was removed.

5.7.3 Replacing sample probe and fill port

TIP: It is recommended to replace the sample probe and the fill port at the same time as the main tubing harness is replaced (see section 5.6.1: Exchanging main tubing harness).



Always wear gloves! Danger of infection!



*To make sure that the pump does not turn on during this procedure, press **NO** until [**MAINTENANCE?**] is displayed. Press **YES** and [**CLEAN?**] will appear. Then, do not press any keys.*

- Open the front door.
- Open the sample door.
- Remove the needle carefully from the holder. Unplug the tubing.
- Reattach the new needle to the tubing and snap the needle back into the holder.



Fig. 5-23

Then replace the fill port as follows:

- Remove the fill port holder from the sample probe mechanism.
- Carefully remove the fill port from the fill port holder. Unplug the tubing.
- Mount the new fill port into the fill port holder in the reverse way and attach the tubing.
- Mount the fill port holder at it's default position.
- Close the sample door and the front cover.
- Press **NO** until [**PERFORM CALIBRATION?**] will be displayed.
- Press **YES** to start the calibration.



Dispose of the sample probe and the fill port according to local regulations (hazardous waste!).

5.7.4 Replacing printer paper

TIP: *The printer paper is heat sensitive on one side only. Please make sure that you insert the paper roll correctly. (See figure 5-24)*

The thermal printer paper contains an indicator strip to alert you when the paper roll should be changed. To change the roll:

- Open the front cover of the analyzer.
- Remove the remaining paper by pressing the paper feed button.
- Place a new paper roll in the paper tray and thread it into the feeder slot.
- Press the paper feed button to completely feed the paper through the printer.



Fig. 5-24



Fig. 5-25

TIP: *Pressing the paper advance button once, advances the paper 10 lines.*

- Close the front cover and tear off any excess paper.
- Cut the end of the roll to make a square edge.



Do not pull the paper out of the printer to avoid damage to the printer.

6

6 *Troubleshooting*

6.1	Error messages	6-1
6.2	Printer Error Codes.....	6-5
6.3	Service functions	6-5
6.3.1	Testing the electrodes	6-6
6.3.2	Testing the sample sensor.....	6-7
6.3.3	Testing the ISE Pack sensor	6-7
6.3.4	Testing RFID Sensor.....	6-7
6.3.5	Testing the sample door.....	6-8
6.3.6	Testing the pump	6-8
6.3.7	Testing the valves	6-8
6.3.8	Testing the interface	6-9
6.3.9	Testing the amplifier.....	6-9
6.4	Configuration codes.....	6-10
6.4.1	Entering service codes.....	6-10
6.4.2	Removing service codes.....	6-10
6.4.3	List of service codes.....	6-10

6 Troubleshooting


Your InSight Electrolyte Analyzer is designed to provide long, trouble-free service. However, any scientific measuring device may occasionally malfunction, requiring the operator to identify the cause of the problem.

The following information lists displayed messages and recommends steps that should help you return your InSight Electrolyte Analyzer to operation. If your InSight Electrolyte Analyzer does not perform correctly after conducting the basic steps outlined in this chapter, you should contact your supplier for technical assistance.

6.1 Error messages

Message	Cause	Action
Status: Not calibrated	Calibration has been interrupted	<ul style="list-style-type: none"> Perform system calibration to return the instrument to the [H-READY] mode
Standard A not found	As the analyzer aspirates a sample of Standard A, the solution is detected by the sample sensor which must detect its presence without encountering any air bubbles. If the sample sensor is unable to properly detect the presence of Standard A solution in the programmed time period, the error message is displayed.	<ul style="list-style-type: none"> Check the fluid remaining in the ISE Pack. If less than 5% remains, replace the ISE Pack. Check for clots or crystals that may have formed in the Standard A tubing or the electrode chamber Replace the ISE Pack. Ensure the sample sensor is securely plugged in and perform the sample sensor test to ensure that the sensor is operating correctly. If necessary, clean the sample sensor. Replace the peristaltic pump tube set to ensure correct aspiration of Standard A
Standard B not found	If Standard A is aspirated properly, but Standard B is not detected, check Standard B tubing for crystallization or leaks. If no crystallization or leaks are found, see Action.	<ul style="list-style-type: none"> Replace the ISE Pack. PRIME first.
Standard C not found (Li activated)	Detection of Standard C aspiration is the same as for Standard B.	<ul style="list-style-type: none"> Replace the ISE Pack. PRIME first.
Inspect Sample Sensor	The sample sensor is calibrated with air during each calibration. For proper functioning, the sensor must provide a reading of 80 to 120 units when air is detected. To check sample sensor response, perform sample sensor test. Open the sample door and press the YES button to aspirate various fluids through the sensor. For transparent fluids (e.g. water), the reading should increase by at least 40 units. For blood samples (not transparent), the reading should decrease by at least 40 units. The pump can be stopped at any time by pressing NO	<ul style="list-style-type: none"> Clean sample sensor by using the Electrode Maintenance procedure Check for correct drying of the measuring chamber during the wash cycle. Replace the peristaltic pump tube set. Perform a calibration to remove the error message.

Message	Cause	Action
Inspect Reference Housing	When the analyzer fails to detect a flow of Reference Solution into the measuring chamber, this message will be displayed. This test is performed at the beginning of each calibration cycle	<ul style="list-style-type: none"> • Check for proper filling of the reference housing • Ensure that the reference tubing is securely connected to the receptacle. • Since this test utilizes Standard A, make sure that it is aspirated properly into the measuring chamber. If not, replace the ISE Pack. • Clean the reference housing. (See chapter 5.4: "Monthly maintenance").
Please shut sample door	This message occurs when the sample door is not closed within 20 seconds after the sample is in place, or when the door has been opened and no sample is detected.	<ul style="list-style-type: none"> • Close the sample door within time allowed
Sample not found	<p>Under certain conditions, the analyzer may not detect the presence of a sample and an error message will occur:</p> <ul style="list-style-type: none"> • sample door remaining open more than 20 seconds after samples are in place • air bubbles in the sample • a sample volume too small to analyze • no sample being aspirated through the sampling mechanism. <p>To abort a sample, close the sample door during sample aspiration.</p>	<ul style="list-style-type: none"> • Repeat the sample to see if it is detected properly on a second trial. • Close the sample door within time allowed. • Check sample aspiration and look for the presence of clots in the sampling system. • Check for the presence of o-rings and for proper sealing of the electrodes. • Ensure that the sample sensor is plugged in and perform the Sample Sensor Test to verify that it is operating correctly. • Replace pump tubing.
Cleaning solution not found	<p>Under certain conditions, the analyzer may not detect the presence of cleaning solution:</p> <ul style="list-style-type: none"> • Air bubbles in the cleaning solution • Too small volume of cleaning solution • The cleaning solution not being aspirated properly through the sampling mechanism 	<ul style="list-style-type: none"> • Check for presence of o-rings and proper sealing of the electrodes. • Check for aspiration of cleaning solution and look for presence of clots in the sampling system. • Check for the proper sealing of the pump windings. • Ensure that the sample sensor is plugged in and perform the Sample Sensor Test to verify that it is operating correctly
Conditioning solution not found	<p>Under certain conditions, the analyzer may not detect the presence of conditioning solution:</p> <ul style="list-style-type: none"> • Air bubbles in the conditioning solution • Too small volume of conditioning solution • The conditioning solution not being aspirated properly through the sampling mechanism 	<ul style="list-style-type: none"> • Check for presence of o-rings and proper sealing of the electrodes. • Check for aspiration of conditioning solution and look for presence of clots in the sampling system. • Check for the proper sealing of the pump windings. • Ensure that the sample sensor is plugged in and perform the Sample Sensor Test to verify that it is operating correctly

Message	Cause	Action
Printer jam or defect	If the printer attempts to print to the paper and the paper jams in the feeder, this message will be displayed briefly, then the sample results will appear on the display	<ul style="list-style-type: none"> First turn analyzer off, then remove the printer by placing your index finger behind the back edge of the printer module and pull it forward. Remove the jammed paper and re-insert the printer into the analyzer. <i>Tip: To ease removal of jammed paper, the printer head can be moved by turning the spindle gear on the left side.</i> <p> Paper jams should be cleared as soon as possible to avoid damaging the printer</p>
Inspect electrodes	<p>If none of the electrodes passes the calibration:</p> <ul style="list-style-type: none"> a stable reading of standard A could not be obtained within 6 aspirations of Standard A the slope A-B or A-C is outside of the limits. 	<ul style="list-style-type: none"> Ensure that electrodes are properly plugged in. Check for proper operation of the reference electrode assembly. If necessary, clean the reference electrode housing or replace the reference electrode. Perform daily maintenance. Replace the ISE Pack
Clog inspect fluid path	<p>If the unit is unable to clear the sample path or to aspirate any of the 3 Standards at the beginning of a calibration:</p> <ul style="list-style-type: none"> the cause: a defective reference housing (increased discharge of KCL) 	<ul style="list-style-type: none"> Make sure that the electrode O rings are present and the seated properly. Make sure there are no leaks in the fluid path. Check for clog or crystallization in the fluid path: especially in the sample probe, the tubing to the sample sensor and in the sample sensor. Ensure that the sample sensor is securely plugged in and perform a sample sensor test to ensure the sensor is operating correctly. If necessary, clean the sample sensor. Replace the reference electrode housing.

Message	Cause	Action
↑↑↑↑ ↓↓↓↓	In case the unit displays arrows up or arrows down instead of the sample results, the concentration of the sample is outside of the measurement range (see chapter 2: “Specifications”).	<ul style="list-style-type: none"> • In the case of a urine sample, arrows up instead of the K^+-result indicate that further dilution is necessary (see chapter 3.3.4: “Urine samples”). • Check for Proper sample preparation (see chapter 3). • Check for small air bubbles in the sample after aspiration into the sample chamber. • Check for proper aspiration of standard A.
↑↓	With Ca^{2+} activated, the calibration report will print an arrow up or down instead of the actual temperature, if the temperature is measured out of range (Range 10.0°C - 40°C). The temperature sensor is located in the right side electrode holder.	<ul style="list-style-type: none"> • Ensure the sample sensor cable is securely plugged in. • Make sure room temperature is within specified limits (15°C to 32°C/60°F to 90°F). • Perform the Amplifier Test to measure the actual temperature. • With the front door opened, the temperature displayed should be approx. 5°C above the actual room temperature.
Err.	If this message is displayed instead of the sample results, no valid voltage reading could be obtained from the electrode.	<ul style="list-style-type: none"> • Check for proper sample preparation (see chapter 3). • Check if electrode holder is locked in the back position. • Check for proper filling of the reference housing. • Check for air bubbles in the sample.
Change ISE Pack	When the monitored fluid level in the ISE Pack reached 5% remaining, the analyzer will automatically print [Change ISE Pack] at the end of each report. This message is present only in printed form.	<ul style="list-style-type: none"> • Replace the ISE Pack (see chapter 5.7.2: ”Checking reagent fluid level and changing the ISE Pack).
Invalid Fluid Pack	Fluid pack is not authentic.	<ul style="list-style-type: none"> • Ensure pack barcode is correctly entered • Go to [Verify Pack] under [Operator Settings] and reenter the pack barcode. • Ensure pack is properly inserted. • Ensure RFID tag is detected by the instrument. Under [Service Menu], perform [CHECK RFID SENSOR] • Contact Customer Support
Fluid Pack is empty	Pack used to its maximum capacity.	<ul style="list-style-type: none"> • Insert new pack
Expired Fluid Pack	Inserted fluid pack has passed expiry date..	<ul style="list-style-type: none"> • Insert new pack
Duplicate Fluid Pack or Invalid Fluid Pack	Inserted pack not valid.	<ul style="list-style-type: none"> • Contact Customer Support

Message	Cause	Action
**Li Calculation not possible	The lithium result can only be calculated for a Na ⁺ range: 95 – 180 mmol/L (89.6 – 169.5 mmol/L for blood/serum) If the Na ⁺ value of the sample is outside of this range, this message will be displayed and printed instead of the Li ⁺ value.	
Interface error	The interface test (see chapter 6.2.8: "Testing the interface") was not successful.	<ul style="list-style-type: none"> • Check if pins 2 and 3 of the serial port were connected properly. • If the error persists, contact Technical Support.
Check Temp (Ca ²⁺ activated)	This message is printed at the end of the sample report and displayed while the measurement is in process in case the temperature of the sample is outside the range (Range 10°C- 40°C).	<ul style="list-style-type: none"> • Check for proper sample preparation (see chapter 3). • Ensure the sample sensor cable is securely plugged in. • Make sure room temperature is within specified limits (15°C-32°C/ 60°F- 90°F). • Perform the Amplifier Test to measure the actual temperature. • With the front door opened, the temperature displayed should be approx. 5°C above the actual room temperature.

6.2 Printer error codes

The InSight allows for a list of error codes to be printed easily from the **[PRINT REPORTS?]** menu. When in this menu, select **NO** until **[PRINT ERROR CODES?]** is shown and then press **YES**. Here the number of error codes printed can be selected. Press **NO** to scroll through the digits and **YES** to select the desired number. Upon confirming the number entered is correct, the InSight will begin printing the error codes.

6.3 Service functions

The InSight Electrolyte Analyzer has various built-in functions that you can access to evaluate the performance of the instrument. Starting at the [H-READY] display, press **NO** until [SERVICE MENU?] is displayed, and enter **YES**.

6.3.1 Testing the electrodes

The voltage levels of the electrodes can be tested by using standard solutions or an external sample.

- Press **NO**, until [CHECK ELECTRODES?] is displayed.
- Press **YES** and the prompt [CHECK STD A?] will be displayed.
- Press **YES** and the electrode voltages will be displayed.
- Record the values.
- To exit this function, press **NO**.
- The prompt [CHECK STD B?] will be displayed.
- Press **YES** and the electrode voltages will be displayed.
- Record the values.
- To exit this function, press **NO**.
- The prompt [CHECK STD C?] will be displayed.
- Record the values.
- Press **YES** and the electrode voltages will be displayed.
- Compare the measured values to the following table:

Electrode	Standard A		Standard B		Standard C		Allowed difference			
							A-B		A-C	
Na ⁺	-600	to +2400	-1600	to +2000	-600	to +2400	+250	to +680	-50	to +50
K ⁺	-700	to +1000	-2500	to +500	-700	to +1000	+470	to +1200	-40	to +40
Cl ⁻	-3100	to -100	-1000	to +3000	-3100	to -100	-860	to -370	Unused	
Ca ²⁺	-3100	to +1000	-2300	to +2500	-3100	to +1000	-660	to -350	-150	to +150
Li ⁺	-3100	to +1900	-3600	to +1400	-2600	to +3400	+1	to +760	-1730	to -285

TIP: If the voltage difference A-B or A-C is outside the allowable difference, perform daily maintenance or replace the electrode.

- After testing the electrodes against the standard solutions, the InSight Electrolyte Analyzer will prompt [CHECK SAMPLE?]. Press **YES**.
- The instrument will prompt [LIFT DOOR LOAD SAMPLE].
- After introducing the sample, the analyzer will display [THANK YOU] shortly, followed by the electrode voltages.
- Check for stable readings.
- Press **NO** to exit and after a short [PLEASE WAIT] message, the prompt [CONTINUE ELECTRODE TEST?] will appear.
- Press **NO** to exit or to perform other service functions.

6.3.2 Testing the sample sensor

The dry sample sensor should indicate a value of 80-120, aspiration of a clear fluid should increase this value by at least 40 Units.

- Press **NO**, until [**CHECK SAMPLE SENSOR?**] is displayed.
- Press **YES** and the analyzer will display the current sample sensor reading:

DRY: 80-120: 99
YES=PUMP/NO=EXIT

- Press **YES** introduce an external sample through the probe. The display will be like:

EXTERNAL: 236
PUMP: NO=OFF

- Press **NO** to exit and after a short [**PLEASE WAIT**] message, the prompt [**TEST ISE PACK SENSOR?**] will appear.

6.3.3 Testing the ISE Pack sensor

- Press **NO**, until [**CHECK PACK SENSOR?**] is displayed.
- Press **YES** and the current ISE Pack sensor status will be displayed:

FLUID PACK:I
I=IN O=OUT

- Remove the ISE Pack to test the ISE Pack sensor.



If the ISE Pack has been removed during this test, a calibration should be performed to prime all fluid lines.

- Press **NO** to exit or to perform other service functions.

6.3.4 Testing the RFID sensor

- Press **NO**, until [**CHECK RFID SENSOR?**] is displayed.
- Press **YES**.
- Insert the Fluid pack with RFID, and press **Yes** to initiate a scan. RFID tag status will be displayed.
TAG FOUND – Tag Present and RFID Sensor is working.
TAG NOT FOUND – Tag is missing or RFID sensor not working.
- Press **NO** to exit or to perform other service functions.

6.3.5 *Testing the sample door*

- Press **NO**, until [**CHECK SAMPLE DOOR?**] is displayed.
- Press **YES** and the current status will be displayed:

SAMPLE DOOR: C
C=CLOSED O=OPEN

- Open the sample door to see the changes of the status.
- Press **NO** to exit or to perform other service functions.

6.3.6 *Testing the pump*

- Press **NO**, until [**PUMP TEST?**] is displayed.
- Open the front cover and press **YES** to begin the pump test. The analyzer will display:

---PUMP TEST!---

The following tests will be performed and displayed:

- Very low speed
- Low speed
- Medium speed
- Fast speed

An audible change should be heard at the pump's speed change.

- Close the front cover and press **NO** to exit or to perform other service functions.

6.3.7 *Testing the valves*

- Press **NO**, until [**CHECK PINCH VALVES?**] is displayed.
- Press **YES** and the following will be displayed:

TEST VALVE A:
YES/NO = ON/OFF

- Press **YES** to move the valve to the open position.
- Press **NO** to move the valve back to the closed position.
- Press **NO** once more to test the next valve.
- After [**CHECK VALVE R**] is finished, press **NO** to exit or to perform other service functions.

6.3.8 Testing the interface

Press **NO**, until [**CHECK INTERFACE?**] is displayed.

Jump pins 2 and 3 of the serial interface making sure not to short the jumper to ground.

Press **YES**.



Be sure not to short pin 2 or pin 3 to ground, otherwise the interface may get damaged.

The analyzer will display [**CHECK INTERFACE?**] and tries to send some characters and checks if they are received within a set time period.

Check that there was no interface error.

Press **NO** to exit or to perform other service functions.

6.3.9 Testing the amplifier

- Press **NO**, until [**CHECK AMPLIFIER?**] is displayed.
- Press **YES** and the analyzer will display the current amplifier values:

Na=#### K=####
Cl/Ca/Li=####

Press **NO** and the ground voltage will be displayed:

GND =0
-2500mV=-2499

Press **NO** and the temperature will be displayed:

TEMP(mV) =-####
TEMP(°C) = ##.#

Press **NO** to exit or to perform other service functions.

GND =0
-2500mV=-2499

Press **NO** and the temperature will be displayed:

TEMP(mV) =-####
TEMP(°C) = ##.#

Press **NO** to exit or to perform other service functions.

6.4 Configuration codes

The [ENTER CONFIG CODE?] menu allows the programming of various functions into the InSight Electrolyte Analyzer.

6.4.1 Entering service codes

- Starting at the [H-READY] display, press **NO** until [SERVICE MENU?] is displayed, and enter **YES**.
- Press **NO** until [ENTER CONFIG CODE?] is displayed.
- The prompt [ENTER PASSWORD] will appear.

Enter the service code as follows:

- Press **NO**, until the first character appears.
- Press **YES** and the cursor will advance to the second position.
- Repeat this procedure for the other characters.
- Pressing **YES** after the third character will activate the service code.

TIP: If the desired character was missed, keep pressing the **NO** key until the correct character appears. If an incorrect code is entered during the process, the analyzer will prompt [INVALID PASSWORD RETRY?]. Press **YES** to correct the code or **NO** to exit.

TIP: Configuration codes can be printed by entering PSC.


6.4.2 Removing service codes

To remove a service code, enter the codes in the off column. To clear all service codes at one time, enter CSC (Clear Service Codes).

6.4.3 List of service codes

On	Off	Action
HRS	SRH	The resolution of all parameters in blood, plasma and serum samples can be increased by 1 digit in this configuration. QC and standard samples are always shown in high resolution. Urine samples are always shown in low resolution.
ECM	MCE	This configuration will automatically put the instrument into Standby if no samples are run between two automatic calibrations, a 4 hour interval. This feature helps to conserve reagents. When the unit automatically enters the Standby, it is always necessary to perform a calibration to get back to sample testing mode [H-READY].
TDV	VDT	The measured voltages for each electrode are printed on sample and calibration reports when in this configuration. This code should only be used by trained service technicians. This configuration can be turned off by either entering VDT or turning the instrument off and back on.
MGC	CGM	This configuration reports Ca^{2+} blood, plasma and serum values in mg/dL instead of mmol/L. The unit also automatically converts the normal ranges high and low values to mg/dL. QC and standard measurements are always reported in mmol/L even when the instrument is configured as MGL. This facilitates comparison of measured QC Ca^{2+} values to MISSION CONTROLS assay values.

TBO	TBO	The beeper is turned OFF.
QCF	QCF	When the instrument is configured in QCF mode, the QC measurements will be reported in direct ISE values and no correlation factors are applied. When QCF configuration is turned off, the QC results will be reported as flame equivalent values with correlation factors applied. Note: The default condition is FCQ.
LCR	LCR	The lease counter number is printed.
PSC	PSC	All configuration codes are printed.

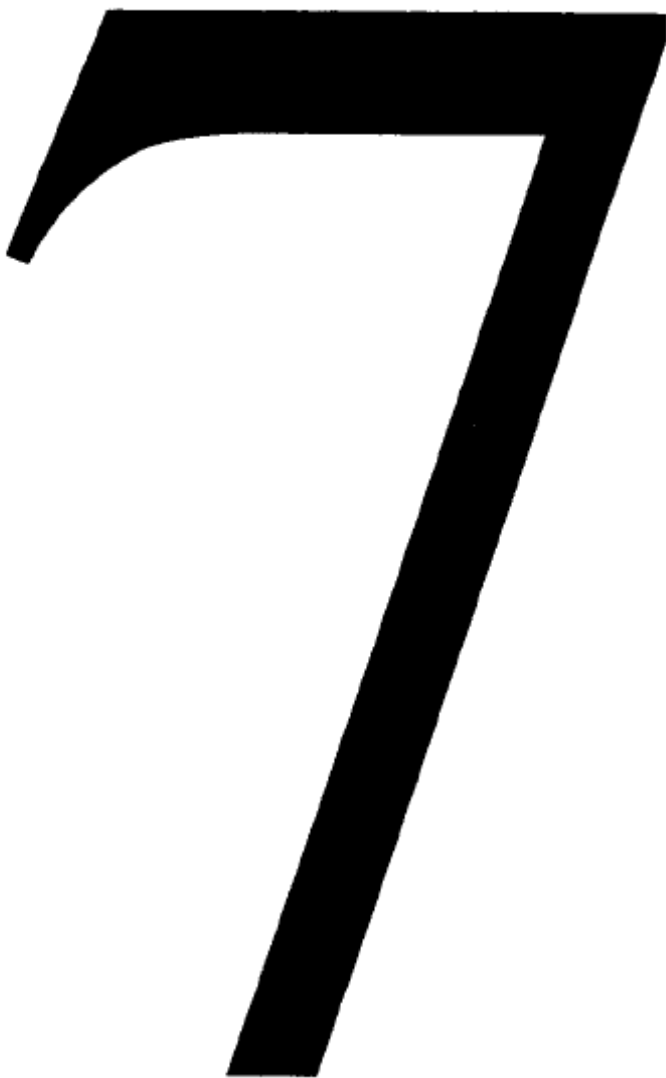

7 *Theoretical foundations*

7.1	Principles of Operation.....	7-1
7.1.1	The measurement principle.....	7-1
7.1.2	Physical principle.....	7-2
7.2	Electrode specifications.....	7-4
7.2.1	Sodium electrode.....	7-4
	Construction	7-4
	Use and care.....	7-4
7.2.2	Potassium electrode.....	7-5
	Construction	7-5
	Use and care	7-5
7.2.3	Chloride electrode.....	7-6
	Construction	7-6
	Use and care	7-6
7.2.4	Calcium electrode	7-7
	Construction	7-7
	Use and care	7-7
7.2.5	Lithium electrode.....	7-8
	Construction	7-8
	Use and care.....	7-8
7.2.6	Reference electrode assembly	7-9
	Construction	7-9
	Reference electrode housing.....	7-9
	Use and care	7-9
7.2.7	Reference electrode Assembly	7-10
	Construction.....	7-10
	Reference Electrode.....	7-10
	Use and care.....	7-10
7.3	Calibration procedure	7-10
7.3.1	ISE calibration	7-10

7 Theoretical foundations

7.1 Principles of Operation

7.1.1 The measurement principle

The InSight Electrolyte Analyzer is a sophisticated medical instrument that uses the Ion Selective Electrode (ISE) measurement principle to precisely determine electrolyte values. Although the technology itself is quite complicated, understanding how the instrument performs sampling analysis is relatively simple. Basically, the analyzer compares an unknown value against a known value to compute the sample's electrolyte level.

An ion-selective membrane undergoes a specific reaction with the type of electrolyte contained in the sample. The membrane is an ion exchanger, reacting to the electrical change of the ion causing a change in the membrane potential, or measuring voltage, which is built up in the film between the sample and the membrane.

A galvanic measuring chain within the electrode determines the difference between the two potential values on either side of the membrane.

The galvanic chain is closed through the sample on one side by the reference electrode, the reference electrolyte and the "open terminal". The membrane, inner electrolyte and inner electrode closed the other side (see Fig. 7-1).

The different ion concentrations between the inner electrolyte and the sample cause an electro-chemical potential to form on the membrane of the active electrode. The potential is conducted by the inner electrode to the input of an amplifier. The reference electrode is connected to ground as well as the second input of the amplifier.

Since the reference electrode is on ground, amplification of the electrode potential allows further signal processing.

The ion concentration in the sample is then determined and displayed by using a calibration curve determined by two measured points of standard solutions with precisely known ion concentrations (two-point calibration), and by using the measured voltage of the sample and the Standard-A (one-point calibration).

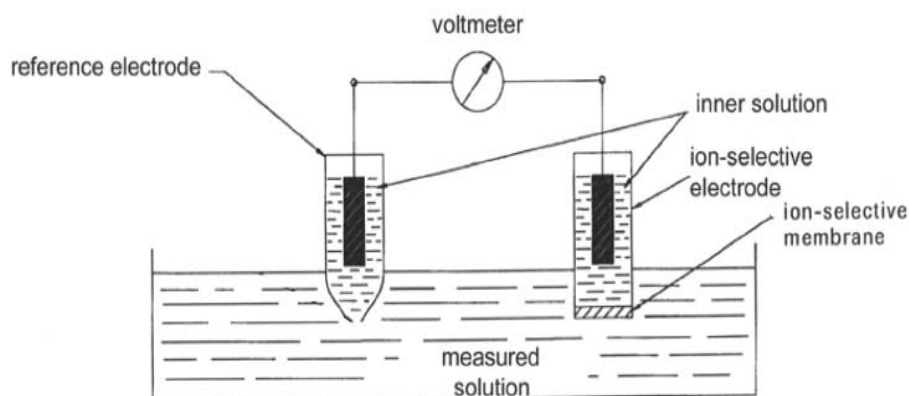


Fig 7-1

7.1.2 Physical principle

An ion-selective electrode is connected with a reference electrode to form a measuring system (see Fig. 7-2). When immersed in a solution that contains the relative ion, the Nernst equation applies:

$$1. E = E' \pm [(R \cdot T) / (n \cdot F)] \cdot \ln a_i$$

or

$$2. E = E' \pm [(R \cdot T) / (n \cdot F)] \cdot [\ln (f_i \cdot c_i)]$$

(+) for cations; (-) for anions

The equation can also be written

$$3. E = E' \pm S - \log (f_i \cdot c_i)$$

E	the measured electrical potential
E'	the e.m.f. of the system in a standard solution
a	activity of the ion measured
R	the general gas constant (8.31 J/Kmol)
T	temperature
n	valence of the measured ion
F	Faraday constant (96.496 A.s/g)
f	the activity coefficient
c	the concentration of the measured ion
S	the slope of the electrode

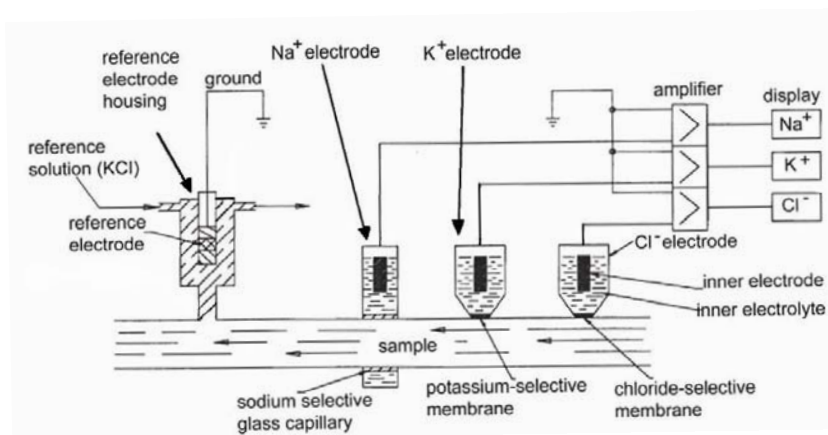


Fig. 7-2

If the ion concentration of one measuring solution is known, the ion concentration of the sample can be determined on the basis of the difference of two measured potentials.

$$4. E_{sample} = E' + S \cdot \log (f_i \cdot c_{i, sample})$$

$$5. E_{standard} = E' + S \cdot \log (f_i \cdot c_{i, standard})$$

$$6. \Delta E = E_{sample} - E_{standard} = S \cdot \log \frac{c_{i, sample}}{c_{i, standard}}$$

 ΔE

the difference between the measured potentials of the sample and the standard

S

the potential difference of the electrode, determined from the potential difference of two measured standard solutions

 $c_{i, sample}$

concentration of the measured ions in the sample

 $c_{i, standard}$

concentration of the measured ions in the standard solutions

The unknown ion concentration in the sample can now be determined by:

$$7. c_{i, sample} = c_{i, standard} \cdot 10^{\Delta E/S}$$

As demonstrated by these equations, the ion selective electrodes do not measure the ion concentration but the activity of the ions concerned. This activity is a criterion of the ion's ability to interact with other ions, in which each ion binds a proportion of its energy.

The ion concentration is calculated on the basis of the ion activity. The correlation is affected by the total number of ions in the solution. Since sodium is the predominant ion in whole blood and serum, the known value of the sodium concentration makes it possible to ascertain and adjust for total ion effect and strength.

7.2 *Electrode specifications*

7.2.1 *Sodium electrode*

The sodium electrode is a glass capillary electrode used for in-vitro diagnostic measurement of sodium ions present in fluid samples. It is designated with a Na⁺ marking on the top surface of the housing.

Construction

Electrode housing:	transparent acrylic plastic
Measuring capillary:	sodium selective glass
Electrolyte chamber:	filled with electrolyte solution for Na ⁺ electrodes
Pin connector:	silver, silver chloride (Ag / AgCl)

Use and care

Sodium electrodes are manufactured for use in InSight Electrolyte Analyzers.

Proper care should be used in handling and storage of the electrode. Never use strong or abrasive cleaners such as alcohol or amphyll on the electrode since these will attack the plastic housing.

Store the electrode in a clean, dry place only after the electrode has been cleaned and rinsed with distilled water and dried with a lint-free cloth.

The o-ring seal should be installed in the electrode during storage.

7.2.2 Potassium electrode

The potassium electrode is a membrane electrode used for in-vitro diagnostic measurement of potassium ions present in fluid samples. It is designated with a K^+ marking on the top surface of the housing.

Construction

Electrode housing:	transparent acrylic plastic
Measuring membrane:	potassium ion selective
Electrolyte chamber:	filled with electrolyte solution for K^+ electrodes
Pin connector:	silver, silver chloride (Ag / AgCl)

Use and care

Potassium electrodes are manufactured for use in InSight Electrolyte Analyzers.

Proper care should be used in handling and storage of the electrode. Never use strong or abrasive cleaners such as alcohol or amphyll on the electrode since these will attack the plastic housing.

Store the electrode in a clean, dry place only after the electrode has been cleaned and rinsed with distilled water and dried with a lint-free cloth.

The o-ring seal should be installed in the electrode during storage.

7.2.3 Chloride electrode

The chloride electrode is a membrane electrode used for in-vitro diagnostic measurement of chloride ions present in fluid samples. It is designated with a Cl^- marking on the top surface of the housing.

Construction

Electrode housing:	transparent acrylic plastic
Measuring membrane:	chloride ion selective
Electrolyte chamber:	filled with electrolyte solution for Cl^- electrodes
Pin connector:	silver, silver chloride (Ag / AgCl)

Use and care

Chloride electrodes are manufactured for use in InSight Electrolyte Analyzers.

Proper care should be used in handling and storage of the electrode. Never use strong or abrasive cleaners such as alcohol or amphyll on the electrode since these will attack the plastic housing.

Store the electrode in a clean, dry place only after the electrode has been cleaned and rinsed with distilled water and dried with a lint-free cloth.

The o-ring seal should be installed in the electrode during storage.

7.2.4 Calcium electrode

The calcium electrode is a membrane electrode used for in-vitro diagnostic measurement of calcium ions present in fluid samples. It is designated with a Ca^{2+} marking on the top surface of the housing.

Construction

Electrode housing:	transparent acrylic plastic
Measuring membrane:	calcium ion selective
Electrolyte chamber:	filled with electrolyte solution for Ca^{2+} electrodes
Pin connector:	silver, silver chloride (Ag / AgCl)

Use and care

Calcium electrodes are manufactured for use in InSight Electrolyte Analyzers.

Proper care should be used in handling and storage of the electrode. Never use strong or abrasive cleaners such as alcohol or amphy on the electrode since these will attack the plastic housing.

Store the electrode in a clean, dry place only after the electrode has been cleaned and rinsed with distilled water and dried with a lint-free cloth.

The O-ring seal should be installed in the electrode during storage.

7.2.5 *Lithium electrode*

The lithium electrode is a membrane electrode used for in-vitro diagnostic measurement of lithium ions present in fluid samples. It is designated with a Li^+ marking on the top surface of the housing.

Construction

Electrode housing:	transparent acrylic plastic
Measuring membrane:	lithium ion selective
Electrolyte chamber:	filled with electrolyte solution for Li^+ electrodes
Pin connector:	silver, silver chloride (Ag / AgCl)

Use and care

Lithium electrodes are manufactured for use in InSight Electrolyte Analyzers.

Proper care should be used in handling and storage of the electrode. Never use strong or abrasive cleaners such as alcohol or amphyll on the electrode since these will attack the plastic housing.

Store the electrode in a clean, dry place only after the electrode has been cleaned and rinsed with distilled water and dried with a lint-free cloth.

The o-ring seal should be installed in the electrode during storage.

7.2.6 Reference electrode housing (Reference Electrode Assembly)

The reference electrode assembly is a device used as an electrical junction between the sample and electrical ground.

Construction

The reference electrode assembly consists of two parts: the reference electrode housing and the reference electrode.

Reference electrode housing

In the reference electrode housing, reference electrolyte solution establishes the electrical contact between the reference electrode and the sample. At the beginning of each measurement, reference electrolyte is pumped into the housing. At the same time a capillary allows a small amount of reference electrolyte to pass into the measuring capillary, thus establishing electrical contact between the sample and the reference electrode.

Use and care

Reference electrode housings are manufactured for use in InSight Electrolyte Analyzers.

Proper care should be used in handling and storage of the electrode housing. Never use strong or abrasive cleaners such as alcohol or amphyll on the housing.

Store the Reference Electrode Housing in a clean, dry place only after the Reference Electrode has been removed and the housing cleaned and rinsed with distilled water and dried with a lint-free cloth.

The o-ring seals should be installed in the housing during storage.

7.2.7 Reference electrode (*Reference Electrode Assembly*)

The reference electrode completes the electrical circuit between the reference electrolyte and electrical ground.

Construction

This is accomplished by a cotton wool (saturated with reference electrolyte)-calomel (Hg_2Cl_2)-mercury (Hg)-platinum wire-connecting pin junction

Use and care

Reference electrodes are manufactured for use in InSight Electrolyte Analyzers.

Proper care should be used in handling and storage of the reference electrode. Never use strong or abrasive cleaners such as alcohol or amphy on the electrode.

Store the reference electrode in the transport housing provided with the electrode at the time of purchase. Make sure that the transport housing is filled with reference electrolyte solution (you may use the reference electrolyte solution remaining in the reference electrode housing at the time of disassembly).




*Never rinse the reference electrode with distilled water!
Never allow the reference electrode to become dry!*

7.3 Calibration procedure

7.3.1 ISE calibration

The InSight Electrolyte Analyzer calibrates Na^+ , K^+ , Ca^{2+} and Cl^- electrodes, using only two aqueous base solutions. Lithium calibration is achieved with a third calibrator in order to make a sodium correction.

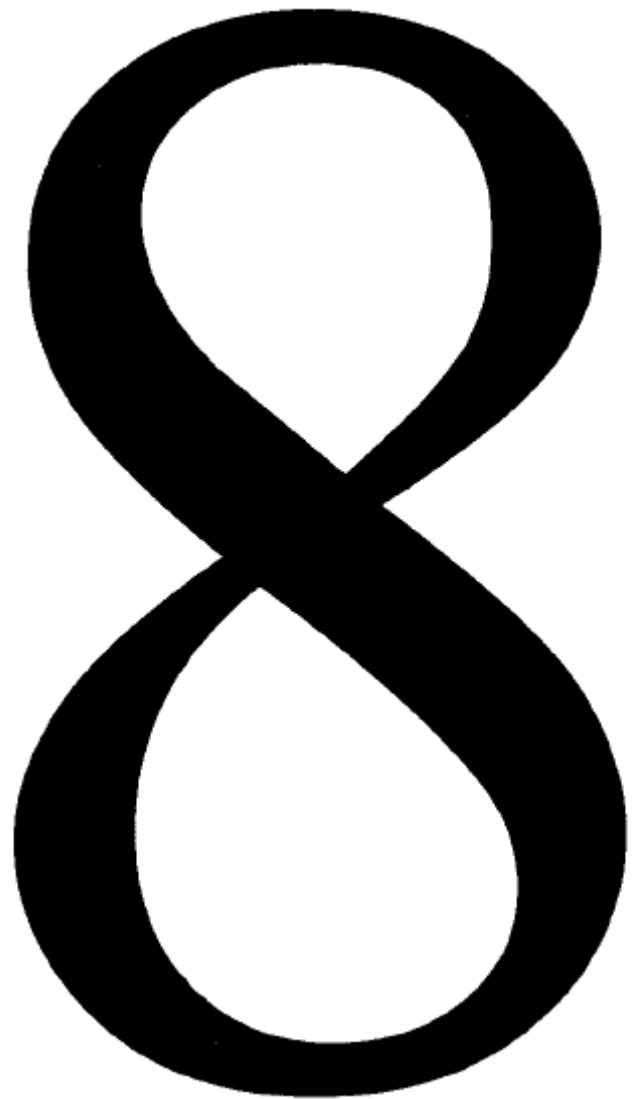

8 *Appendix*

8.1	Description of Reports.....	8-1
8.1.1	Measurement report	8-1
8.1.2	QC report.....	8-1
8.1.3	Calibration report	8-2
8.1.4	Error Report.....	8-3
8.1.5	Configuration Report	8-3
8.1.6	Normal Range Report.....	8-4
8.1.7	Correlation Factors Report	8-5
8.2	ISE Pack.....	8-6
8.3	Worksheet for determination of the correlation coefficients.....	8-7
8.4	Maintenance Schedule Template	8-8
8.5	Program Flow Chart	8-9
8.6	Parts Warranty	8-10
8.7	Vet Mode Specific Instructions.....	8-11
8.7.1	Program Flow Chart.....	8-11
8.7.2	Veterinary, Blood, Serum, Plasma Sample measurement.....	8-12
8.7.3	Veterinary Normal Ranges.....	8-12

8 Appendix

8.1 Description of reports

8.1.1 Measurement report

<p>ISE ANALYZER H-InSight Na-K-Cl 16-JUN-11 10:55 10L01247</p> <p>ID: _____</p> <hr/> <p>TEST: URINE</p> <p>TEST#: 7</p> <p>Na=194 mmol/L K = 13.4 mmol/L Cl = 62 mmol/L</p>
--

Fig 8-1

8.1.2 QC report

<p>ISE ANALYZER H-InSight Na-K-Cl 16-JUN-11 10:38 10L01247</p> <p>TEST: QC 1</p> <p>TEST#: 1</p> <p>Na=109.0 mmol/L K = 2.02 mmol/L Cl = 77.7 mmol/L</p> <p>VALUES ACCEPTED!</p> <p>EXECUTE MAINTENANCE!</p>
--

<p>ISE ANALYZER H-InSight Na-K-Cl 16-JUN-11 10:44 10L01247</p> <p>TEST: QC 2</p> <p>TEST#: 2</p> <p>Na=135.4 mmol/L K = 4.55 mmol/L Cl = 103.2 mmol/L</p> <p>VALUES ACCEPTED!</p> <p>EXECUTE MAINTENANCE!</p>

<p>ISE ANALYZER H-InSight Na-K-Cl 16-JUN-11 10:44 10L01247</p> <p>TEST: QC 3</p> <p>TEST#: 3</p> <p>Na=153.9 mmol/L K = 6.46 mmol/L Cl = 113.8 mmol/L</p> <p>VALUES ACCEPTED!</p> <p>EXECUTE MAINTENANCE!</p>

Fig 8-2

8.1.3 Calibration report

ISE ANALYZER
H-InSight
Na-K-Cl
16-JUN-11 10:33
10L01247

CAL REPORT

MAINTENANCE
LAST DONE:
00-JAN-00 07:00

STD A
Na = 1540mV (3)
K = -465mV (3)
Cl = 184mV (3)

A-B DIFF
Na = 514mV (3)
K = 909mV (3)
Cl = -618mV (3)

Temp = 27.4 ° C

ISE PACK STATUS
93% REMAINING

Fig. 8-3

8.1.4 Error Report

ISE ANALYZER
H-InSight
Na-K-Cl
29-JUN-11 07:29

ERROR REPORT

INSPECT
SAMPLE SENSOR
28-JUN-11 15:45
INVALID CAL
28-JUN-11 15:44
STD B NOT FOUND
28-JUN-11 14:37
STD A NOT FOUND
28-JUN-11 14:31
INVALID CAL
28-JUN-11 14:30
INVALID CAL
28-JUN-11 14:29
INVALID CAL
28-JUN-11 14:29
INSPECT
SAMPLE SENSOR
28-JUN-11 14:28
STD C NOT FOUND
28-JUN-11 14:28
STD B NOT FOUND
28-JUN-11 14:27
STD A NOT FOUND
28-JUN-11 14:26
INSPECT
REF HOUSING
28-JUN-11 14:22
INVALID CAL
28-JUN-11 14:21
STD C NOT FOUND
28-JUN-11 14:20
STD B NOT FOUND
28-JUN-11 14:19
STD A NOT FOUND
28-JUN-11 14:18

8.1.5 Configuration Report

CONFIGURATION:
HRS, TDV

8.1.6 Normal Range Report

ISE ANALYZER
H-InSight
Na-K-Cl
16-JUN-11 10:33
10L01247

NORMAL RANGES
UNITS: [mmol/L]
Na: 136 – 145
K: 3.50-5.10
Cl: 97 – 111

QC RANGES
UNITS: [mmol/L]

LEVEL 1
LOT#

Na: 40 - 200
K : 1.70 - 15.00
Cl: 50 - 200

LEVEL 2
LOT#
Na: 40 - 200
K : 1.70 - 15.00
Cl: 50 - 200

LEVEL 3
LOT#
Na: 40 - 200
K : 1.70 - 15.00
Cl: 50 - 200

8.1.7 Correlation Factors Report

ISE ANALYZER
H-InSight
Na-K-Cl
16-JUN-11 11:03
10L01247

CORR. FACTORS:
Na OFFS. = 0.0
Na Slope = 1.000
K OFFS. = 0.00
K SLOPE = 1.000
Cl OFFS. = 0.0
Cl SLOPE = 1.000

URINE. FACTORS:
Na OFFS. = 0.0
Na Slope = 1.000
K OFFS. = 0.00
K SLOPE = 1.000
Cl OFFS. = 0.0
Cl SLOPE = 1.000

8.2 ISE Pack



Reagents are to be used for in-vitro diagnostic use only.

The ISE Pack contains the following solutions

Standard A

Standard B

Standard C

Reference Solution



Caution: A waste container is provided inside the ISE Pack which contains potentially infectious materials. Handle with appropriate care to avoid skin contact or ingestion!

8.3 Worksheet for determination of the correlation coefficients

TIP: This page can be used as template for duplication.

Parameter:

low concentration/ref. device	
Sample 1	
Sample 2	
Sample 3	
Sum 1,2,3	
$\overline{X}_1 = \frac{\text{Sum 1, 2, 3}}{3}$	

low concentration/CorLYTE	
Sample 1	
Sample 2	
Sample 3	
Sum 1,2,3	
$\overline{y}_1 = \frac{\text{Sum 1, 2, 3}}{3}$	

high concentration/ref. device	
Sample 1	
Sample 2	
Sample 3	
Sum 1,2,3	
$\overline{X}_2 = \frac{\text{Sum 1, 2, 3}}{3}$	

low concentration/ref. device	
Sample 1	
Sample 2	
Sample 3	
Sum 1,2,3	
$\overline{y}_2 = \frac{\text{Sum 1, 2, 3}}{3}$	

$d = \overline{X}_2 - \overline{X}_1$

$c = \overline{y}_2 - \overline{y}_1$

\overline{X}_2	
\overline{X}_1	
$d = \overline{X}_2 - \overline{X}_1$	

\overline{y}_2	
\overline{y}_1	
$c = \overline{y}_2 - \overline{y}_1$	

$m = c / d$

$m = \overline{y}_2 - \overline{y}_1$

d	
c	
$m = c / d$	

m	
\overline{y}_1	
$m \cdot \overline{y}_1$	

$b = \overline{X}_1 - m \cdot \overline{y}_1$

$b = \overline{X}_1 - m \cdot \overline{y}_1$

\overline{X}_1	
$m \cdot \overline{y}_1$	
$b = \overline{X}_1 - m \cdot \overline{y}_1$	


b=

8.4 Maintenance Schedule Template

ISE Electrolyte Analyzer Maintenance Schedule

Month: _____

	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
DAILY																															
Perform cleaning																															
Perform conditioning																															
WEEKLY																															
Clean Sample probe/fill port																															
Clean analyzer surfaces																															
MONTHLY																															
Clean reference electrode housing																															
SEMI-ANNUALLY																															
Change pump tubing																															
ANNUALLY																															
Change main tubing harness																															
UNSCHEDULED MAINTENANCE																															

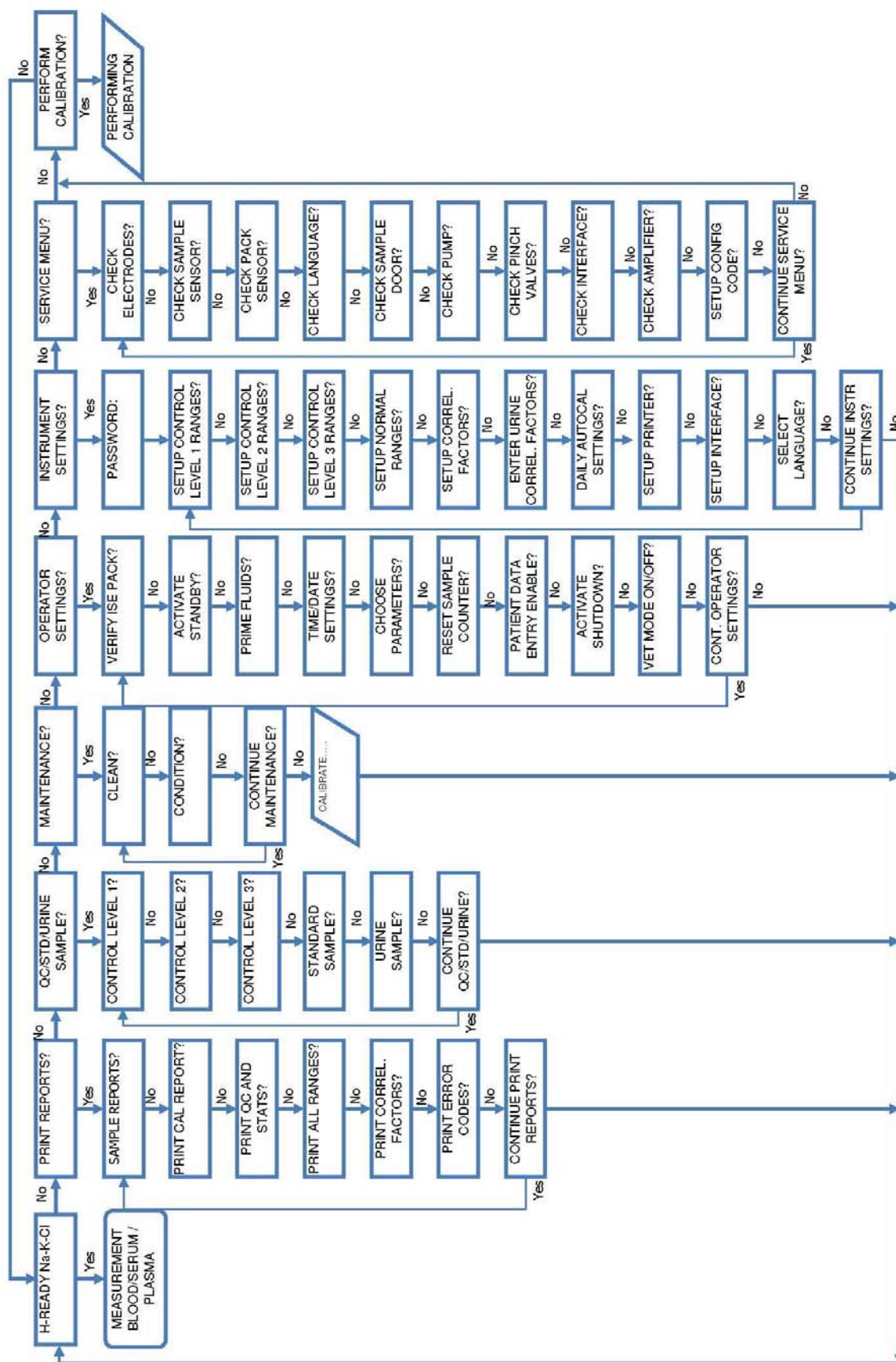


8.6 *Parts Warranty*

Replacement Part #	Description	Warranty
AV-BP0413D	Na ⁺ Electrode	12 months
AV-BP0359D	K ⁺ Electrode	6 months
AV-BP0570D	Cl ⁻ Electrode	6 months
AV-BP0360D	Ca ²⁺ Electrode	6 months
AV-BP0962D	Li ⁺ Electrode	3 months
AV-BP5026D	Reference Electrode	12 months
AV-BP5027D	Peristaltic Pump Tubing	3 months
AV-BP5193D	Pinch Valve Tubing Kit	6 months
AV-BP5186D	ISE PACK	Use before expiration date
AV-BP0380D	Na Electrode Conditioner Solution	Use before expiration date
AV-BP1025D	Cleaning Solution	Use before expiration date
AV-BP0521D	Deproteinizer Solution	Use before expiration date
AV-BP0344D	Urine Diluent	Use before expiration date
AV-BP5025D	Thermal Printer Paper	Replace as necessary
AV-BP5019D	Reference Electrode Housing	Replace as necessary
AV-BP5006D	Sample Probe	Replace as necessary
AV-BP5036D	Sample Sensor	Replace as necessary
DD-0201	Mini Keyboard	Replace as necessary
DD-0202	Barcode Scanner	Replace as necessary

8.7 Vet Mode Specific Instructions

8.7.1 Program Flow Chart

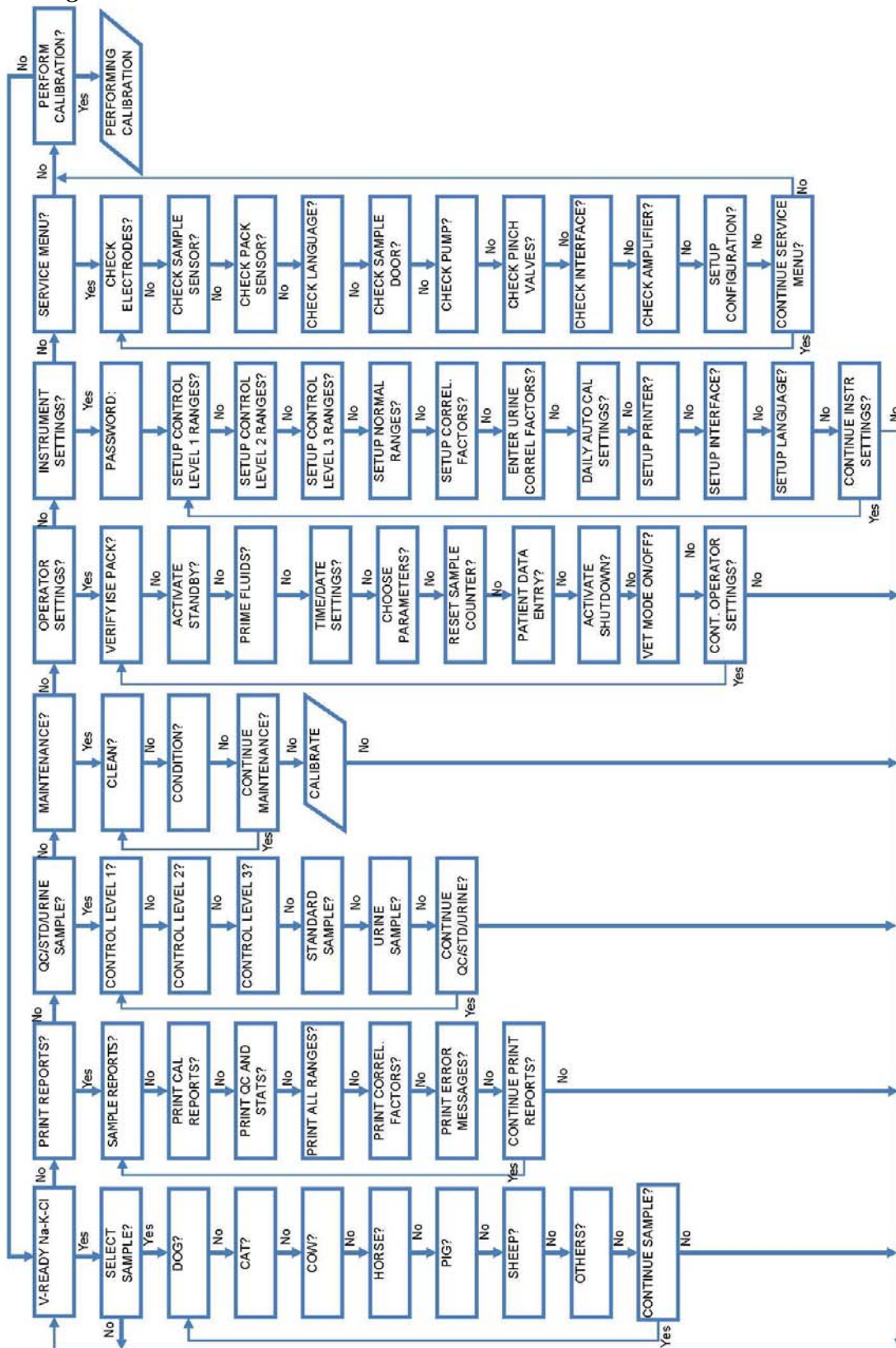

8.7.2 Veterinary Blood, Serum, Plasma Sample measurement

Whenever [V-READY] appears, the analyzer is ready to carry out veterinary blood, serum, plasma and urine measurements.



It is very important that the main door is closed during sampling to provide shielding from sources of electromagnetic interference.



Urine samples require dilution, and must be analyzed in the urine mode. Instructions for analyzing urine samples: see "Urine samples".

- Press **YES** and InSight will prompt you to enter the patient species.
- Press **YES** to select the appropriate species.
- Lift the door and load the sample.
- Hold the sample container under the probe until [**WIPE PROBE_SHUT SAMPLE DOOR**] is displayed.
- Use a lint-free tissue to wipe the probe. Then close the sample door.

8.7.3 Veterinary Normal Ranges

Parameter	Na ¹	K ¹	Cl ¹	Ca
Units	mmol/L	mmol/L	mmol/L	mmol/L
Dog ²	140 - 154	3.8 - 5.6	102 - 117	1.25 - 1.45
Cat ²	146 - 159	3.8 - 5.3	108 - 130	1.15 - 1.35
Cow	135 - 148	4.0 - 5.8	96 - 109	***
Horse	133 - 147	2.8 - 4.7	97 - 110	***
Pig	139 - 153	4.4 - 6.5	97 - 106	***
Sheep	142 - 160	4.3 - 6.3	101 - 113	***

¹Kahn, C., et. al., (Eds.), Merck Veterinary Manual, 9th Ed., 2008, Merck & Co., Inc.

²DiBartola, S.P., Fluid, Electrolyte, and Acid-Base Disorders, 3rd Ed., 2005, Elsevier Saunders

*** Not Available

Values that are higher or lower than the programmed normal range will be indicated by an arrow pointing up or down.

The normal ranges can be adapted to the respective laboratory-specific requirements. The ranges can be changed through the programming menu. Follow the instructions for the [INSTRUMENT SETTINGS?] in chapter 4.2 and enter 3-digit password.

- Press **NO** until [SETUP NORMAL RANGES?] is displayed. Press **YES**.
- Press **YES** when the animal is displayed for which normal ranges will be changed.
- The analyzer will display the normal ranges of the current parameter configuration:
Na LOW=xxx
Na HIGH=xxx ok?
 Normal ranges can be entered by pressing **NO**
- Press **NO** until the desired number is displayed, and press **YES** to advance to the next position. Repeat this for all numbers of the normal range.
- After entering the normal ranges, the new values will be displayed. Verify correctness of the ranges:
Na LOW=xxx
Na HIGH=xxx ok?

TIP: If an incorrect number was entered inadvertently, press **NO**, and the analyzer allows for retyping the normal range.

- Press **YES** and the display will advance to the next parameter.

Repeat the above procedure for all parameters and each animal type. After all activated parameters have been programmed, the display prompts [**SETUP FACTORS FOR CORRELATION?**]. This allows programming of the correlation factors.

TIP: All other procedures for InSight are the same in veterinary mode as in human mode.

9

9 Index

A

Analyzer components.....	1-7
Appendix	8-1

B

Bibliography.....	2-4
-------------------	-----

C

Calibrations	2-5
Calibration Report	8-2
Classification.....	2-10
Cleaning	
Analyzer surfaces	5-4
Reference electrode housing	5-5
Sample probe mechanism	5-4

D

Decontamination	5-1
Description of reports	8-1
Dimensions	2-10
Display.....	1-8
Disposal of	
Electrodes	1-6
Instrument	1-6
Reference Electrode.....	1-6
ISE Pack	1-6

E

Electrical Data.....	2-10
Electrode Specifications.....	7-4
Calcium electrode	7-7
Chloride electrode	7-6
Lithium electrode	7-8
Potassium electrode.....	7-5
Reference electrode	7-10
Reference electrode housing	7-9
Sodium electrode.....	7-4
Entering services codes	6-10
Environmental parameters	2-6
Error Messages	6-1
Exchanging Main tubing harness	5-9
Pump tubing set	5-8

G

General notes.....	1-2
--------------------	-----

I

Installation	1-12
Preparing the analyzer for Operation	1-18
Selecting language	1-19
Interferences.....	2-3
ISE Pack	1-10

K

Keypad	1-8
--------------	-----

L

Limitations.....	2-3
Limitations of clinical analysis	3-3
Electrolytes.....	3-3
General	3-3
List of service codes.....	6-10
Location	1-12

M

Maintenance	
Annual	5-9
Exchanging main tubing harness	5-9
Cleaning reference electrode housing.....	5-5
Daily	5-2
Monthly	5-5
Semi annual	5-8
Exchanging pump tubing set	5-8
Unscheduled maintenance	5-10
Material setup	4-2
Measurement and calibration procedures.....	1-5
Measuring chamber	1-14
Measuring procedure.....	3-4

N

Normal Ranges.....	3-8
--------------------	-----

Testing the interface	6-9
Testing the language switch	6-7

P

Parts Warranty	8-9
Performance parameters	2-1
Peristaltic pump	1-9
Preanalytic	3-1
Acceptable anticoagulants	3-1
Sample collection	3-1
Sample collection containers	3-2
Sample handling	3-2
Sample requirements	3-1
Printer	1-8, 2-11
Printing QC report	4-5
Product Data	2-10

Q

QC	
Material setup	4-2

R

Reagents	2-7
ISE Pack	2-7
QC Material	2-9
Urine diluent	2-9
Rear panel	1-11
Recommended disinfectant	5-2
Removing service codes	6-10
Replacing	
Electrodes	5-10
Printer Paper	5-14
Sample probe	5-13
Reports	
Calibration report	8-2
Measurement report	8-1
QC report	8-1

S

Sample collection	3-1
Sample handling	3-2
Aqueous solutions	3-3
Plasma	3-2
Serum	3-2
Whole blood	3-2
Sample measurement	3-4
Direct ISE	3-6
Urine samples	3-7
Sample probe mechanism	1-9
Sample throughput	2-5
Sample types	2-5
Sample volumes	2-5
Service codes	6-10
Service functions	6-6
Testing the amplifier	6-9
Testing the electrodes	6-6

Testing the pump	6-8
Testing the sample door	6-8
Testing the sample sensor	6-7
Testing the ISE Pack sensor	6-7
Shutdown	1-26
Specifications	2-1
Measurement parameters	2-1
Reproducibility	2-1
Standby mode	1-30
Symbols	1-2
System description	1-7

T

Temperature / humidity / stability	
Electrodes	2-6
Instrument	2-6
QC Material	2-9
Urine diluent	2-9
Theoretical foundations	7-1
Troubleshooting	6-1

V

Valves	1-10
Vet Mode	
Flow Chart	8-8
Normal Ranges	8-11
Sample Measurement	8-11
Selection	3-10