

Troubleshooting/FAQ

Q/ What is the shelf life of the Rapidbac Vet?

A/ 18months

Q/ How do I store the Rapidbac Vet?

A/ The kits can be stored at room temperature

Q/ How do I interpret the results if the line appears faint?

A/ Faint lines are considered valid results, including the control line. A faint line in a positive result may be caused by a low concentration of bacteria or a species of bacteria that expresses low levels of the targeted antigen on its surface. In these cases, bacteriuria is present even though the line is faint.

Q/ A control line has not appeared, what should I do?

A/ Ensure the correct procedure has been followed and repeat the test using a new test strip

