



## The new and improved veterinary glucometer

### *Advantages of the new meter:*

- Improved FAD-GDH enzyme test strip
- Smaller sample volume
- Wider HCT range
- Strip eject
- Single code strip for all species
- Developing Now - Upgradeable option of voice instructions and Bluetooth connectivity

### *Additional key features:*

- Calibrated specifically for cats, dogs & horses
- Biosensor auto code technology
- Accurate Results
- Easy to use blood glucose meter
- Vets can train owners to home test
- Small sample (0.7µl) of whole blood is required
- Large LCD screen
- Accurate results displayed within 5 seconds
- Readable range 1.1 – 33.3 mmol/l
- 500 result automatic memory
- Adjustable alert range alarms
- mmol/l or mg/dl
- HCT range 20 - 60%



Tel: +44 (0)1204 669033

Fax: +44(0)1204 669034

Email: [sales@woodleyequipment.com](mailto:sales@woodleyequipment.com)

Web: [www.woodleyequipment.com](http://www.woodleyequipment.com)

OLD STATION PARK BUILDINGS, ST. JOHNS STREET, HORWICH, BOLTON, LANCASHIRE, BL6 7NY, UK

**WOODLEY**  
EQUIPMENT COMPANY LTD.

## Why use the gPet PLUS veterinary glucose meter rather than a human glucose meter?

Human glucometers are not always accurate when measuring glucose levels in animals. This is because red blood cell size and the distribution of glucose within the blood differs from species to species. A proportion of glucose is contained within the red blood cells and the remainder in the surrounding plasma. Humans have a higher haematocrit compared to cats, dogs and horses, as well as larger red blood cells. Therefore, the distribution of glucose within plasma is proportionally much lower in humans compared to these animals, as a larger portion of glucose is contained within the red blood cells. Subsequently, these differences in glucose distribution have a bearing on the glucose result when testing animals with a meter designed for human use, as false results are reported in some instances.

## Species specific calibration

The gPet PLUS veterinary glucose meter is calibrated specifically for use in cats, dogs and horses. Each gPet PLUS meter contains all relevant calibration information to ensure an accurate result is reported for each species. Vets no longer need to persist with inaccurate human meters as there is now a low cost, veterinary specific alternative.



### Testing Made Easy!

Step 1 Insert the test strip  
(Meter will turn on automatically)



Step 2 Apply Blood



Step 3 Results in 5 seconds



### Kit includes:

- gPet PLUS meter
- AAA battery x 2
- Test strips (10)
- Code strip
- Control solution
- Carry case
- User guide
- Test strip insert
- Control solution insert
- Quick reference guide
- Lancing device
- Lancet x 10

Tel: +44 (0) 1204 669033

Fax: +44 (0) 1204 669034

Email: [sales@woodleyequipment.com](mailto:sales@woodleyequipment.com)

Web: [www.woodleyequipment.com](http://www.woodleyequipment.com)

**WOODLEY**  
EQUIPMENT COMPANY LTD.

OLD STATION PARK BUILDINGS, ST. JOHNS STREET, HORWICH, BOLTON, LANCASHIRE, BL6 7NY, UK