

LipoClear

Lipaemia is the presence of abnormally high levels of lipoproteins in the blood.

The effect of these lipids in samples causes interference in certain analytical methodologies, which results in unreliable results or complete rejection of the sample.

LipoClear is a solution to **remove the lipoproteins** in lipaemic samples, to give a **clear sample** prior to analysis.

Benefits:

- Non-toxic
- Non-ionic
- Non-carcinogenic
- Use fresh or frozen serum
- Easy to use
- Only 0.5mL of the sample is required
- Supplied in pre-dispensed micro-centrifuge tubes



Use:

- Allow sample and LipoClear to reach room temperature
- Add 0.5mL of sample to LipoClear tube
- Mix well and allow to stand for 5 minutes
- Centrifuge as per instructions
- Remove cleared sample for analysis
- Analyse cleared specimens within 1 hour
- To compensate for the dilution, multiply by the appropriate dilution factor

Clinispin Multi-Purpose Centrifuge (MPC)

Woodley's Clinispin MPC is ideal for use with LipoClear pre-dispensed micro-centrifuge tubes, with a centrifuge time of just 2 minutes.

The Clinispin MPC is a **high-speed** centrifuge equipped with a **maintenance free** drive, digital display and simple interface for **silent and efficient operation**.

Features:

- Maintenance Free Drive
- Digital Display
- Easy to Use
- 3 Pre-Defined Pre-sets

(Pre-sets can be varied for each centrifugation if needed)

1. Serum / Plasma Pre-Set (Microtube rotor) - set to 12,000rpm for 2 minutes
2. HCT / QBC® Pre-Set (HCT / QBC® rotor) - set to 12,000 rpm for 5 minutes
3. Urine Pre-Set (Microtube rotor) - set to 1,000rpm for 1 minute



Tel: +44 (0)1204 669033 (Option 1)
Fax: +44(0)1204 669034
Email: sales@woodleyequipment.com
Web: www.woodleyequipment.com



WOODLEY
EQUIPMENT COMPANY LTD.

OLD STATION PARK BUILDINGS, ST. JOHNS STREET, HORWICH, BOLTON, LANCASHIRE, BL6 7NY, UK