

RapidBac™ Vet 2.0

In-Practice UTI Test

Now with a bufferless formulation



TEST FOR UTI

IN JUST 15 MINUTES

ENABLES EARLY TREATMENT.

"PASSIONATE ABOUT LABORATORY DIAGNOSTICS"

WOODLEY
EQUIPMENT COMPANY LTD.

RapidBac™ Vet 2.0

In-Practice UTI Test

RapidBac Vet 2.0 is a lateral flow immunoassay for the detection of Gram-positive and Gram-negative bacteria in Dog and Cat Urine.

Combine with Flexicult® Vet for complete In-Practice Urinalysis


FEATURES

- Bufferless formation
- Test immediately in-practice
- Results in 15 minutes
- Easy to use
- Room temperature storage
- No special apparatus required



SIMPLE TEST PROCEDURE

1. Using the Disposable Pipette, add 5 drops of sample into the test vial.
2. Make sure all the liquid is at the bottom of the Test Vial by swirling it and tapping the Test Vial gently on a flat surface. Wait 5 minutes.
3. Swirl the Test Vial gently again. This step is essential for proper detection.
4. Place the Test Strip into the Test Vial with the arrows pointing down. Wait 10 minutes. Do not disturb the Test Strip or Test Vial during this time.
5. Take the Test Strip out of the Test Vial and interpret the results.

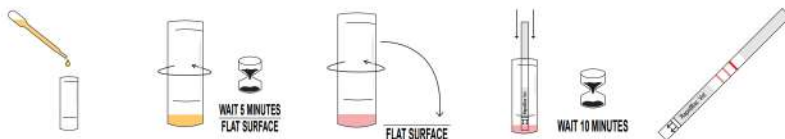

SENSITIVITY AND SPECIFICITY


| | Sensitivity | Specificity | Reproducibility |
|-----|-------------|-------------|-----------------|
| Dog | 97.4% | 98.8% | 100% |
| Cat | 90.9% | 96.6% | 100% |

INTERPRETING RESULTS

Positive Result


Any colour development at Line 1 or Line 2 indicates the presence of bacteria in the sample.

- If both Line 1 and Line 2 are present, a Gram-negative organism has been detected.
- If Line 2 is present and Line 1 is absent, the detected organism may be Gram-positive or Gram-negative.

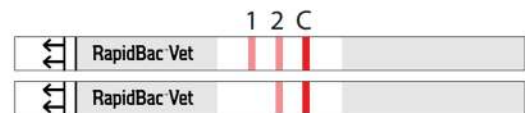

Negative Result

If only the Control Line (C) develops colour - no bacteria detected.

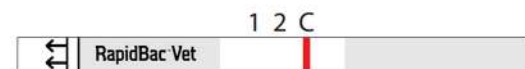

Invalid Result

If the Control Line does not develop colour, the result is invalid - repeat test. In rare samples with high level of Gram-negative bacteria, both Line 1 and 2 appear but the Control line is weak or absent - this result should be interpreted as Positive, Gram-negative organism detected.

+44 (0) 1204 669033 (Option 1)

sales@woodleyequipment.com

www.woodleyequipment.com

Old Station Park Buildings
St. John Street
Horwich
Bolton
Lancashire
BL6 7NY
UK

**"PASSIONATE ABOUT
LABORATORY DIAGNOSTICS"**

**WOODLEY
EQUIPMENT COMPANY LTD.**