

iCup® Urine Drug Test Kit



iCup drug test kits are available in the configurations listed below:

- I-DOA-1237 - 3 panel iCup drug test - THC, COC, mAMP
- I-DOA-3137 - 3 panel iCup drug test - COC, THC, mAMP
- I-DOA-187-041 - 8 panel iCup drug test - THC, COC, MDMA, OPI, PCP, AMP 300, BZO, BAR
- I-DOA-11107 - 10 panel iCup drug test - THC, COC, mAMP, OPI, PCP, AMP, BZO, BAR, MTD, TCA
- I-DOA-1107-051 - 10 panel iCup drug test - mAMP, COC, THC, AMP, OPI, OXY, PPX, BZO, BAR, MDMA


iCup drug test kits with adulteration test strips are available in the configurations listed below:

- I-DUA-147-012 - 4 panel iCup drug test - THC, COC, mAMP, OPI plus adulterant test
- I-DUA-157-013 - 5 panel iCup drug test - THC, COC, mAMP, OPI, AMP plus adulterant test
- I-DUA-157-023 - 5 panel iCup drug test - THC, COC, AMP, OPI, PCP plus adulterant test
- I-DUA-157-034 - 5 panel iCup drug test - THC, COC, OPI, PCP, mAMP plus adulterant test
- I-DUA-167-022 - 6 panel iCup drug test - THC, COC, mAMP, OPI, PCP, AMP plus adulterant test
- I-DUA-167-022 - 6 panel iCup drug test - THC, COC, OPI, AMP, mAMP, BZO plus adulterant test
- I-DUA-187-012 - 8 panel iCup drug test - THC, COC, OPI, AMP, mAMP, PCP, BZO, BAR
- I-DUA-197-013 - 9 panel iCup drug test - THC, COC, mAMP, OPI, PCP AMP, BZO, BAR, MTD plus adulterant test

Version: V2 10/19

Simple Testing Procedure

1 Collect Specimen

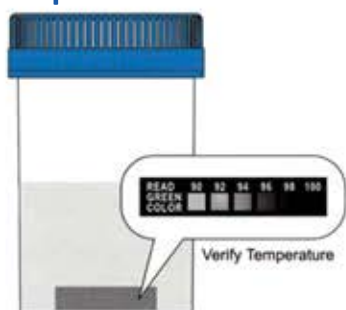


2 Peel Back Label




3 Read the Results



Read results at five (5) minutes.
Results stable up to sixty (60) minutes.

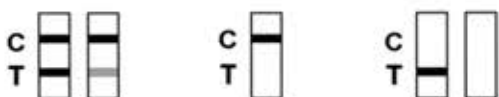

NEGATIVE NON-NEGATIVE INVALID

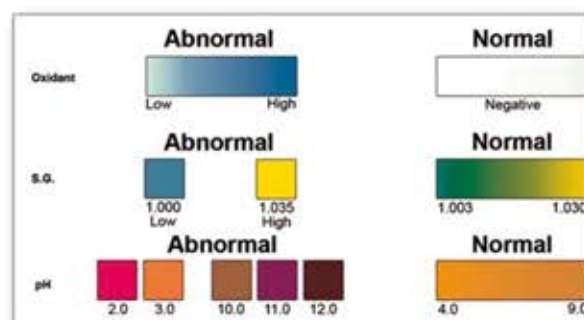

SAMPLE ONLY - Colour chart NOT exact

UK Office

t: +44 (0)8456 777001 f: +44 (0)8456 777002

USA Office

t: 1 800 471 9200 / 1 718 606 0516 f: 1 718 606 0955

e: hello@woodleytrialsolutions.com

www.woodleytrialsolutions.com

woodley
TRIAL SOLUTIONS

Woodley Trial Solutions is a division of Woodley Equipment Company